Speed: 600 rpm - Torque: 1 Ncm

Vitesse: 600 tr/min - Couple: 1 Ncm

Drehzahl: 600 Upm - Drehmoment: 1 Ncm

- Max. 3-4 back and forth gentle strokes.
- Light touch, let the instrument work.
- Work 3-4 seconds in a row, get out, clean the blade and irrigate the canal.

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Prepare glide path first, then

Fill the canals and pulp chamber with irrigant.

1. BR0 - only 4 gentle strokes - clean the flutes.
- Repeat until approximately 4-6mm of coronal part of the canal has been prepared.
- After use of BR0, repeat irrigation.
- Recapitulate to full working length (WL) with a manual file #15.
- Fill the canal and pulp chamber with irrigant.
2. Use BR1 with 4 gentle strokes. If this instrument does not reach the WL, clean the instrument and repeat until the WL is achieved.
3. Use BR2 and BR3 as described for BR1. DO NOT use BR3 to full WL on canals with severe apical curvatures. In this case go to step 1 of BioRaCe Extended Set protocol.
4. Use BR4 and BR5 as explained for BR1-3. In most cases, the final apical preparation is then achieved.
5. For larger canals, (see anatomical chart on www.fkg.ch), go to step 3 of BioRace Extended Set protocol.

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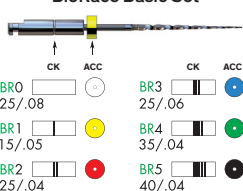
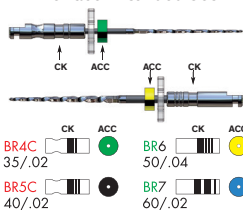
BioRace Extended Set

1 blister includes / contient / enthält: 1 × BR4C, 1 × BR5C, 1 × BR6, 1 × BR7

1. For severe apical curvatures, instruments BR4C and BR5C should be used to prepare the apical canal. If the instrument does not reach the WL with 4 gentle strokes, DO NOT FORCE the instrument. Irrigate the canals and repeat.
2. For complicated curvatures it is recommended to use additional FKG instruments (e.g. S-Apex, ScoutRace or manual SMG files).
3. For larger canals the two additional instruments BR6 and BR7 from BioRace Extended Set can be used as explained for BR1-5.

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NB. Copious irrigation at all times. Cleaning the files after 4 gentle strokes is essential for safe and efficient use of these instruments.

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Example on instruments of 25 / 31 mm

Exemple sur des instruments de 25 / 31 mm

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EN Sterilization protocol for reusable medical devices

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Non sterile products First use

Carefully follow the rules herebelow
Cleaning and Sterilization

Sterile products First use

1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice

Second use:
Carefully follow the rules herebelow
Second and following uses.

Cleaning

1. Manual cleaning with/without Ultrasonic assistance

Thorough cleaning allows efficient disinfection & sterilization.

2. Rinse

Under distilled/demineralized water at least 1 min.
Then dry the instruments.

3. Inspection - Check

Sort out damaged or worn out instruments.

4. Packing

Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

Sterilization

5. According EN/ISO 17664 protocol

Autoclave: 134°C/273°F,
2.2 bar during 18 min.

6. Storage

Keep devices in sterilization wrap/pouch in a dry and clean environment.

 Always refer to manufacturer's instructions for cycle & duration

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Second and following uses

7. Pre-disinfection

Plunge instruments after use in detergent solution and eventually brush them manually.

8. Rinse

Under running water at least 1 min.
Then dry the instruments.

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 Complete operating protocol and warnings on www.fkg.ch

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Pre-/Disinfection

DO NOT USE

- solutions containing:
- phenol (corrosion)
 - aldehyde (blood fixation)
 - di-/triethanolamines (corrosion)

Sterilization with chemiclav or dry air devices have not been validated by FKG Dentaire.
Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

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- Des solutions contenant:
- phenol (corrosion)
 - aldehyde (fixation du sang)
 - di-/triethanolamines (corrosion)

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Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

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A single patient use is recommended to avoid cross-contamination.

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This is to certify that:

Emergo Global Consulting, LLC

2500 Bee Cave Road
Building 1, Suite 300
Austin
Texas
78746
USA

Holds Certificate Number:

FS 594789

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 for the following scope:

Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry.

For and on behalf of BSI:

Graeme Tunbridge, Senior Vice President Medical Devices

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 1 of 2

...making excellence a habit.™

Certificate No: FS 594789

| Location | Registered Activities |
|--|--|
| Emergo Global Consulting, LLC 2500 Bee Cave Road Building 1, Suite 300 Austin Texas 78746 USA | Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry. AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations. |
| I.L. Emergo Israel Ltd. Andrei Sakharov 9 Building 25 - 8th floor Matam North Haifa 3190501 Israel | AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations. |
| Emergo Europe Westervoortsedijk 60 6827 AT, Arnhem The Netherlands | Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation delivered to the medical device industry. |

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 2 of 2

This certificate was issued electronically and remains the property of BSI and is bound by the conditions of contract.

An electronic certificate can be authenticated [online](#).

Printed copies can be validated at www.bsigroup.com/ClientDirectory

Issuing Body: BSI Group The Netherlands B.V., John M. Keynesplein 9, 1066 EP Amsterdam, The Netherlands

BSI Group The Netherlands B.V. is registered in The Netherlands under number 33264284 | A Member of the BSI Group Holdings B.V.

Contact Office: 12950 Worldgate Drive, Suite 800, Herndon, VA 20170-6007 USA.