HYBENX®

Oral Tissue Decontaminant For Professional Use Only

Instructions For Use

INDICATIONS:

HYBENX is indicated for the removal of plaque biofilm, when, in the opinion of the dental professional, an adjunctive treatment for cleaning is required to remove plaque biofilm. HYBENX is a single use product indicated as a topical rinse/solution that is delivered locally in the oral cavity. HYBENX is administered by an irrigation syringe or other means chosen by the dental professional.

DESCRIPTION:

HYBENX contains sulphonated phenolics and sulphuric acid, which are tissue denaturants, in an aqueous solution. HYBENX is an opaque purple, semi-viscous liquid or gel.

DIRECTIONS:

HYBENX is to be used topically only with an irrigation syringe or other means chosen by the dental professional, to the gingival tissues or periodontal pockets surrounding the affected tooth/teeth, dental pulp or the pulpal canal for endodontic treatment. Treatment should start with the deepest portion and to be irrigated thoroughly to the more superficial areas. Excess fluids reduce the effectiveness of HYBENX. Use cotton or compressed air to dry tissues prior to application. For sites with excess bleeding, introduce a generous amount of HYBENX directly at the source of the bleeding to displace the blood. The product should be applied for 10 to 20 seconds. Remove HYBENX by thoroughly irrigating the oral cavity with water and evacuate with high speed suction. Removal of the product is required to stop the cleaning process. Properly dispose of any remaining product.

NOTE: If the patient has not been given a local anesthetic the product may elicit a stinging sensation during application until it is rinsed off. Incidental contact with any oral tissue will cause it to turn "white" until the tissue rehydrates itself.

HYBENX is contraindicated for use with calcium hydroxide paste and similar related products. HYBENX is intended to be used in its original semi-viscous state. Diluting with water and/or combining HYBENX with another product will render the product ineffective and potentially harmful.

If using HYBENX during Etching procedures, HYBENX should only be applied and removed prior to application of the etching solutions.

IRRIGATION TIP:

Depending on the clinician's preference, a blunt-locking tip needle or a closed-end side-port needle may be used for product application and irrigation. Only the blunt-locking tip is enclosed.

WARNINGS:

Keep out of sight and reach of children.

Do not use if the patient is allergic to a material that contains sulphur in any form.

Because of its nature, prolonged use on normal tissue should be avoided. HYBENX will eventually slough any tissue to which it is applied if excess volume is used and an application time is grossly extended.

It is important to use high speed evacuation/suction to limit overflow of any excess material to areas that do not need to be treated.

PRECAUTIONS:

HYBENX is for use in the oral cavity only.

Avoid eye contact. If eye exposure occurs, immediately remove any lenses and irrigate eyes for at least 15 minutes with luke warm water.

If skin contact occurs, immediately rinse thoroughly with water. HYBENX will damage clothes or fabric.

Safety and effectiveness in pregnant women and children under the age of 12 has not been established.

ADVERSE REACTIONS:

HYBENX may cause local irritation and swelling upon administration. It is important to irrigate liberally with water and evacuate with high speed suction. If excess irritation occurs discontinue use.

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©2015 EPIEN Medical, Inc.