Instructions for Use

Self-Adhesive Calcium Releasing Base-Liner

GENERAL INFORMATION TheraBase Ca is a dual-cure, chemically bonding, self-adhesive base-liner...

Warnings: Avoid contact with the skin, unpolymerized (meth)acrylate resins may cause skin sensitization in susceptible persons...

Precautions: Working and setting times will be accelerated in the warm insaroral environment.

1100 W. Irving Park Road Schaumburg, IL 60193 U.S.A. 1-847-634-6000 1-800-244-3368

BISCO France 208, allée de la Coudolette 13680 Lançon de Provence France 33-4-90-42-98-92

INSTRUCTIONS FOR USE Preparation: If a pulp exposure occurs, BISCO recommends using TheraCa LC as a pulp capping agent.

Mixing Procedures: 1. Remove the syringe cap on the dual syringe. 2. Dispense a small amount of material onto a mixing pad...

HELPFUL HINTS TheraBase Ca set in a mixing pad may not appear to set. This is due to oxygen inhibition of the polymerization process.

HYGIENE: Use of commonly available hygienic protective covering to avoid any contamination of the TheraBase Ca dual syringes during the treatment is recommended.

DISPOSAL: Refer to community provisions relating to waste. In their absence, store all normal or regional provisions relating to waste.

WARRANTY: BISCO Inc. recognizes its responsibility to accept liability for any damages or loss, either direct or consequential, resulting from the use of the product.

NOTICE: Report any serious incident to BISCO and the competent authority of the EU member state in which the user and/or patient is established.

TO REORDER ACCESSORIES: X-81270P - 20 gauge auto-mixing syringe with dual syringe BISCO, Inc.

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TECHNISCHE INFORMATIONEN Working Time: Minimum 1 minute (includes mixing) at 22°C (71.6°F).

INSTRUCTIONS FOR USE Preparation: If a pulp exposure occurs, BISCO recommends using TheraCa LC as a pulp capping agent.

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ZUR BESTELLUNG VON ZUBEHÖR: X-81270P - 20-gauge Automischer Zylinder mit Doppelspritze BISCO, Inc.