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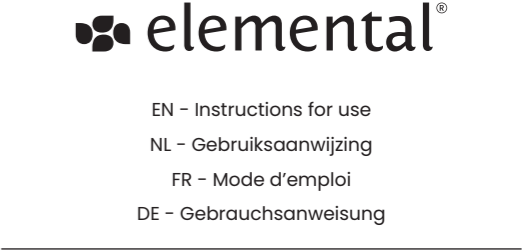
EN – Elemental Granulate – Instructions for use

Hydroplastic, bacteriostatic granulate for supramucosal universal dental use as mechanical coverage of oral surgical sites, non-resorbable.

RECOMMENDED INDICATIONS

Use Elemental Granules for mechanical coverage of oral post-surgical sites and making custom splints:

- Covering of post-surgical sites
- Covering of palatal donor sites
- Covering of periodontal wounds
- Covering & stabilization of hemostatic agents
- Covering & stabilization of membranes

CONTRAINDICATIONS

Elemental Granulate is not suitable for use in patients under 18 years of age. Not suitable for submucosal use or use in direct contact with wounds with secondary healing. Not suitable for use longer than 30 days. The product may shrink (<1%) which may cause retention to deteriorate over time.

DIRECTIONS FOR USE

Dosing: determine the amount of granules needed for the desired application in advance. Indications for quantities can be consulted in the practical protocols. The volume of the granules hardly changes after heating. If necessary, granules can be added to the material as long as the mass is in a moldable state to gradually reach the desired amount.

Heat up water or sterile saline to approximately 100 degrees celsius by using an applicable heating source. Add the Elemental Granulate to the water/sterile saline and stir with a non-plastic tool until the Elemental Granulate clumps together and becomes soft & mouldable. This takes 10 to 15 seconds. Remove the clumped Elemental Granulate from the water / sterile saline and control manually whether the material is a) not too hot to be placed in the mouth and

EN - Elemental Granulate – Instructions for use

Thermoplastic, bacteriostatic granulate for supramucosal universal dental use as mechanical coverage of oral surgical sites, non-resorbable.

RECOMMENDED INDICATIONS

Use Elemental Granulate for mechanical coverage of oral post-surgical sites and making custom splints:

- Covering of post-surgical sites
- Covering of palatal donor sites
- Covering of periodontal wounds
- Covering & stabilisation of hemostatic agents
- Covering & stabilisation of membranes

CONTRAINDICATIONS

Elemental Granulate is not suitable for use in patients under 18 years of age (United States: not suitable for use in patients under 22 years of age). Not suitable for submucosal use or use in direct contact with wounds with secondary healing. Not suitable for use longer than 30 days.

DIRECTIONS FOR USE

Dosing: determine the amount of granules needed for the desired application in advance. Indications for quantities can be consulted in the practical protocols. The volume of the granules hardly changes after heating. If necessary, granules can be added to the material as long as the mass is in a moldable state to gradually reach the desired amount.

Heat up water or sterile saline to approximately 90°C or 194°F by using an applicable heating source. Add the Elemental Granulate to the water/sterile saline and stir with a non-plastic tool until the Elemental Granulate clumps together and becomes soft & mouldable. This takes 10 to 15 seconds. Remove the clumped Elemental Granulate from the water / sterile saline and control manually whether the material is a) not too hot to be placed in the mouth and b) is moldable enough to manipulate with your hands. (attention: the material might stick slightly to some types of gloves, latex gloves are recommended)

If the Elemental Granulate is too hot, wait a couple of seconds until it has cooled slightly. If it is not mouldable enough, or if the granules do not mould into a homogenous mass, reheat the material in boiling water / sterile saline and repeat the moulding process. Cooler water may result in faster setting times and less moldability.

Make sure the site is completely covered. To avoid irritation in the mouth, check if the stent has no sharp edges. If necessary, reheat the edges of the material to soften the sharp edges. Working and curing times depend on the amount and temperature of the material used. Always ensure good retention in the mouth. By reheating the material, you can adjust or completely reshape the molded material.

After placement of the material, perform a follow-up on your patient to verify that they are not experiencing any discomfort, including irritation from sharp edges or altered bite position or occlusion.

PRODUCT INFORMATION

Elemental Granulate is thermoplastic, bacteriostatic, nonresorbable. Material consistency is plastic-like and highly moldable after heating. Hardening starts 30 to 60 seconds after taking it away from the heat source. Hardens into a rigid material after approximately 90 seconds. Individual granules may slightly differ in size.

CAUTION

Verify that after applying & setting, the material has sustainable mechanical retention or other fixation before leaving the patient. The material may experience shrinkage (<3%) after setting and during the application time, which may lead to mild deformation and loss of retention. PPE (Personal Protective Equipment) should always be worn. The material might stick slightly to some types of gloves, latex gloves are recommended. Under no circumstances whatsoever is it allowed to re-use the Elemental Granulate on other patients, or on the same patient after a long (60 minutes or more) period of time. Always use fresh unused Elemental Granulate for new patients.

Do not heat up the Elemental Granulate to temperatures over 150°C or 302°F as this may damage the product's inherent properties. A slight discoloration of the material might take place in the patient's mouth due to food consumption or use of antiseptics.

In rare cases, the use of Elemental Granulate may cause patient discomfort due to suboptimal retention. If you cannot obtain durable retention or fixation of the material, discontinue use of Elemental Granulate immediately.

Elemental Granulate contains small concentrations of zinc citrate, which can be absorbed by the body in minimal, non-toxic amounts due to degradation of the material. If your patient loses the Elemental Granulate application or notices any discomfort from wearing it, instruct them to stop use immediately if possible and come in for a check-up visit.

Do not use Elemental Granulate if the seal inside the box has been damaged. Do not use Elemental Granulate if you suspect any product defects.

RECOGNIZING PRODUCT DEFECTS

Please contact ZINKH NV in case you suspect any product defect.

Indications of a faulty product may include: *

- Insufficient binding of the Elemental Granulate into a moldable paste
- Discoloration of the Elemental Granulate before usage
- Insufficient hardening of the Elemental Granulate after cooling off to body temperature
- Damage to product seal

*Please consider this list as indicative and non-exhaustive.

STORAGE

Keep Elemental Granulate away from direct sunlight. Conserve Elemental Granulate in a dry area between 5-30°C or 40-85°F. Do not use for more than 36 months after opening the package.

Undesired effects - Reporting: If you become aware of any kind of adverse reaction, reaction or similar events occurring while using this product, including those not mentioned in these instructions for use, please report it immediately to vigilance@withelemental.com, as well as to the competent authority of the country in which you are located.

For any questions, requests or complaints, please contact us directly at:
ZINKH NV, Zuiderlaan 1-3 (box 8), 9000 Ghent, Belgium, +32 9 241 58 80

Last revised: 29/04/2024. Manufactured and distributed by Zinkh NV, Zuiderlaan 1-3, 9000 Gent