# Instructions for use

## Golden rules

Recommended speed: 800 rpm  
Torque: 1.5 Ncm

- Glide path should be established before using BT-Race sequence (min. size ISO 15/.02).
- All files should be used with long and gentle pecking motion (3-4 back and forth gentle strokes).
- Copious irrigation throughout the procedure.

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## For most cases

Speed: 800 rpm (600-1000 rpm)

1. After the coronal access is attained, the WL should be obtained with small hand files (ISO 06, 08, 10 or 15) depending the constriction of each canal.
2. A glide path should be performed with small stainless steel or NiTi files up to ISO 15 before using BT-Race sequence.
3. Files of the entire sequence should be used to full WL before changing to the next file in the sequence. Per file, total working time in one canal should not exceed 10 seconds.
4. Use the sterile BT1 with a long and gentle pecking motion (3-4 back and forth strokes). If BT1 does not reach WL, clean the instrument, irrigate and repeat until the WL is achieved.
5. Recapitulate with K-File ISO 15 to remove the debris and keep the glide path open, irrigate.
6. Use sterile BT2 up to WL in the same manner as BT1.
7. Recapitulate with K-File ISO15, irrigate.
8. Use sterile BT3 up to WL in the same way as BT1.

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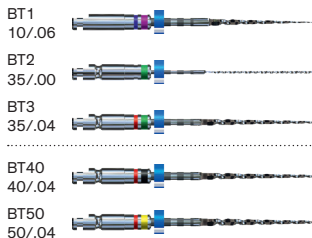
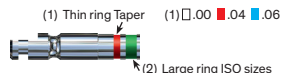
## For larger apical sizes

Speed: 800 rpm (600-800 rpm)

BT3 is used for minimal biological apical preparation.

For larger apical preparations use BT40 or BT50.

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This is to certify that:

**Emergo Global Consulting, LLC**

2500 Bee Cave Road  
Building 1, Suite 300  
Austin  
Texas  
78746  
USA

Holds Certificate Number:

FS 594789

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 for the following scope:

Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry.

For and on behalf of BSI:

Graeme Tunbridge, Senior Vice President Medical Devices

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 1 of 2

...making excellence a habit.™

Certificate No: FS 594789

Location	Registered Activities
Emergo Global Consulting, LLC 2500 Bee Cave Road Building 1, Suite 300 Austin Texas 78746 USA	Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry. AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations.
I.L. Emergo Israel Ltd. Andrei Sakharov 9 Building 25 - 8th floor Matam North Haifa 3190501 Israel	AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations.
Emergo Europe Westervoortsedijk 60 6827 AT, Arnhem The Netherlands	Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation delivered to the medical device industry.

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 2 of 2

This certificate was issued electronically and remains the property of BSI and is bound by the conditions of contract.

An electronic certificate can be authenticated [online](#).

Printed copies can be validated at [www.bsigroup.com/ClientDirectory](http://www.bsigroup.com/ClientDirectory)

Issuing Body: BSI Group The Netherlands B.V., John M. Keynesplein 9, 1066 EP Amsterdam, The Netherlands

BSI Group The Netherlands B.V. is registered in The Netherlands under number 33264284 | A Member of the BSI Group Holdings B.V.

Contact Office: 12950 Worldgate Drive, Suite 800, Herndon, VA 20170-6007 USA.