VISTA | APEX

## Best-Etch™

37% Phosphoric Acid Etching Gel

REF 502115\* REF 502235\*

REF 502230\* REF 502200\*

\* Asterisk denotes CE marking

## Velvet-Etch™

40% Phosphoric Acid Etching Gel

REF 503315 REF 503300

### Rx Only

Made in China

#### Emergo Europe

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

#### Inter-Med, Inc.

2200 South St.  
Racine, WI 53404, USA  
Tel: +1-262-636-9755

[www.vistaapex.com](http://www.vistaapex.com)

502115-I-V-MUL (1)

**REF****Catalogue  
Number**

**EN** Catalogue number  
**BG** Каталоген номер  
**CZ** Katalogové číslo  
**DA** Katalognummer  
**ET** Kataloogi number  
**FR** Référence catalogue  
**DE** Bestellnummer  
**EL** Αριθμός καταλόγου  
**HU** Katalógusszám  
**IT** Numero di catalogo  
**LV** Kataloga numurs  
**LT** Katalogo numeris  
**PT** Número de catálogo  
**RO** Număr de catalog  
**SK** Katalógové číslo  
**ES** Número de catálogo  
**SV** Katalognummer

**EN** Temperature limitation  
**BG** Ограничение на температурата  
**CZ** Omezení teploty  
**DA** Temperaturbegrænsning  
**ET** Temperatuuri piiramine  
**FR** Limite de température  
**DE** Temperaturbegrenzung  
**EL** Περιορισμός θερμοκρασίας  
**HU** Hőmérséklet korlátozás  
**IT** Limiti temperatura  
**LV** Temperatūras ierobežojums  
**LT** Temperatūros apribojimas  
**PT** Limitação de temperatura  
**RO** Limitarea temperaturii  
**SK** Obmedzenie teploty  
**ES** Límites de temperatura  
**SV** Temperaturgräns

**Follow  
Instructions  
For Use**

**EN** Follow instructions for use  
**BG** Следвайте инструкциите за употреба  
**CZ** Postupujte podle návodu k použití  
**DA** Følg brugsanvisningen  
**ET** Järgige kasutusjuhiseid  
**FR** Suivez les instructions d'utilisation  
**DE** Gebrauchsanweisung befolgen  
**EL** Ακολουθήστε τις οδηγίες χρήσης  
**HU** Kövesse a használati utasításokat  
**IT** Attenersi alle istruzioni per l'uso  
**LV** Izpildiet lietošanas instrukcijas  
**LT** Vadovaukitės naudojimo instrukcijomis  
**PT** Siga as instruções de uso  
**RO** Urmați instrucțiunile de utilizare  
**SK** Postupujte podľa návodu na použitie  
**ES** Siga las instrucciones de uso  
**SV** Följ bruksanvisningen

**Caution,  
Consult  
Accompanying  
Documents**

**EN** Caution, consult accompanying documents  
**BG** Внимание, вижте придружаващите документи  
**CZ** Pozor, prostudujte si přírodní dokumenty  
**DA** Forsigtig, læs de medfølgende dokumenter  
**ET** Ettevaatus, tutvuge kaasasolevate dokumentidega  
**FR** Attention, consulter les documents joints  
**DE** Vorsicht, Begleitdokumente beachten  
**EL** Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα  
**HU** Vigyázat, olvassa el a kísérelő dokumentumokat  
**IT** Attenzione, consultare la documentazione allegata  
**LV** Uzmanību, skatiet pavaddokumentus  
**LT** Atsargiai, skaitykite pridedamus dokumentus  
**PT** Cuidado, consulte os documentos anexos  
**RO** Atenție, consultați documentele însoțitoare  
**SK** Pozor, pozrite si sprírodnú dokumentáciu  
**ES** Precaución: consulte documentos adjuntos  
**SV** Lakttag försiktighet, konsultera medföljande dokument

Do not use  
if package is  
damaged

- EN** Do not use if package is damaged  
**BG** Не използвайте, ако опаковката е повредена  
**CZ** Nepoužívejte, pokud je obal poškozen  
**DA** Må ikke anvendes, hvis pakningen er beskadiget  
**ET** Árge kasutage, kui pakend on kahjustatud  
**FR** Ne pas utiliser si l'emballage est endommagé  
**DE** Nicht verwenden, wenn die Verpackung beschädigt ist  
**EL** Μην το χρησιμοποιείτε εάν η συσκευασία είναι κατεστραμμένη  
**HU** Ne használja, ha a csomagolás sérült  
**IT** Non utilizzare se la confezione è danneggiata  
**LV** Nelietot, ja iepakojums ir bojāts  
**LT** Nenaudokite, jei pakuotė pažeista  
**PT** Não use se o pacote estiver danificado  
**RO** Nu utilizați dacă ambalajul este deteriorat  
**SK** Nepoužívajte, ak je obal poškodený  
**ES** No utilice el producto si el envoltorio está dañado  
**SV** Får inte användas om förpackningen är skadad

Authorized  
Representative  
in the  
European  
Community

- EN** Authorized Representative in the European Community  
**BG** Упълномощен представител в Европейската общност  
**CZ** Zplnomocněný zástupce v Evropském společenství  
**DA** Autoriseret repræsentant i EU  
**ET** Volitatud esindaja Euroopa Ühenduses  
**FR** Représentant agréé pour la Communauté européenne  
**DE** Autorisierte Vertretung in der Europäischen Gemeinschaft  
**EL** Εξουσιοδοτημένος Αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα  
**HU** Az Európai Közösség meghatalmazott képviselője  
**IT** Rappresentante autorizzato nella Comunità Europea  
**LV** Pilnvarotais pārstāvis Eiropas Kopienā  
**LT** Įgaliojotas atstovas Europos bendrijoje  
**PT** Representante Autorizado na Comunidade Europeia  
**RO** Reprezentant autorizat în Comunitatea Europeană  
**SK** Splnomocnený zástupca v Európskom spoločenstve  
**ES** Representante autorizado en la Comunidad Europea  
**SV** Auktoriserad representant i EU

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Irritants/  
Sensitizers/  
Other Hazards

EN	Irritants / Sensitizers / Other Hazards
BG	Дразнещи / Сензитизатори / Други опасности
CZ	Dráždivé látky / senzibilizatory / jiná nebezpečí
DA	Lokalirriterende stoffer / Sensibilisatorer / Andre farer
ET	Ärritjad / Sensibilisaatorid / Muud ohud
FR	Irritants / Sensibilisants / Autres dangers
DE	Reizstoffe / Sensibilisatoren / Sonstige Gefahren
EL	Ερεθιστικά / Ευαισθητοποιητές / Άλλοι Κίνδυνοι
HU	Iritáló anyagok / Szenzibilizáló szerek / Egyéb veszélyek
IT	Irritanti/Sensibilizzanti/Altri pericoli
LV	Kairinoši / Sensibilizatori / Citi apdraudējumi
LT	Dirginančios medžiagos / Jautrinančios medžiagos / Kiti pavojai
PT	Iritantes / sensibilizantes / outros perigos
RO	Iritanți / Sensibilizatori / Alte pericole
SK	Dráždivé látky / senzibilizatory / iné riziká
ES	Irritantes / Sensibilizadores / Otros peligros
SV	Irriterande ämnen / Andra faror / Sensibiliseringsmedel

Corrosion

EN	Corrosive
BG	Корозивен
CZ	Korozivní
DA	Ætsende
ET	Söövitav
FR	Corrosive
DE	Ätzend
EL	Διαβρωτικός
HU	Maró hatású
IT	Corrosiva
LV	Kodīgs
LT	Korozinis
PT	Corrosiva
RO	Coroziv
SK	Žieravý
ES	Corrosiva
SV	Frätande

### Product Description:

Best-Etch™ is a 37% phosphoric acid etching gel with optimum thixotropic flow properties. It is used to etch tooth substances during the enamel-etch and total-etch techniques.

Velvet Etch™ is a 40% phosphoric acid etching gel with optimum thixotropic flow properties. It is used to etch tooth substances during the enamel-etch and total-etch techniques.

### Indications for Use:

Best-Etch™ and Velvet Etch™ are designed to etch and prepare dentin and enamel.

### Directions for Use - Preparation:

- For 1.2ml syringe
  - Remove luer cap.
  - Securely attach desired tip (22ga pre-bent tip recommended).
  - Verify flow prior to intraoral use.
- For 30mL Prefilled Syringe
  - Attach one end of the docking port to the 30mL prefilled syringe and attach the other end of the docking port to an empty 1.2mL syringe.
  - Slowly depress 30mL prefilled syringe plunger while guiding 1.2ml syringe plunger to desired fill. Do NOT overfill syringe.
  - Separate syringes from the docking port and re-cap 30mL prefilled syringe.
  - Securely attach desired tip (22ga pre-bent tip recommended) to 1.2mL syringe.
  - Verify flow prior to intraoral use.

NOTE: To avoid cross contamination, use new syringes and tips for each patient.

### Directions for Use -

#### Direct Etching Process:

- Prepare the tooth.
- Isolate tooth (teeth). The use of a dental dam is recommended.
- Rinse the cavity well with water and dry with oil-free pressurized air.

- For near pulp exposures, place a protective base prior to placing etchant.
- Verify etchant flow prior to intraoral application.
- Apply the etching gel on the enamel edges of the cavity and, where necessary, to the dentine. Lightly agitate the etchant during its application to enhance its action. Recommended application time:
  - 15 - 30 sec. on enamel
  - 10 - 15 sec. on dentine
- Rinse for at least 20 sec. with water.
- If using a bonding adhesive, follow the manufacturers' instructions to determine if the surface of the tooth is to remain moist or if it should be thoroughly dried.

#### Directions for Use -

##### Restorative Cleaning Process:

- Apply the etching gel to the inside of the porcelain veneer or crown for 5-10 seconds.
- Rinse and dry with oil and water-free compressed air.

##### Storage:

- Store between 2°C – 30°C (36°F – 86°F).

##### Warnings & Precautions:

- For dental professional use only.
- Avoid contact with soft tissue (skin, oral mucosa, eyes, etc.). If etching gel touches soft tissue, rinse the area immediately with plenty of water and if necessary, contact a doctor.
- Do not use after the expiration date.
- To avoid cross contamination, do not re-use tips. Disinfect and wipe syringe with disinfectant between uses.
- Always replace syringe cap after use.

Note: For SDS please go to [www.vistaapex.com](http://www.vistaapex.com)

May be corrosive to metals. Causes severe skin burns and eye damage. Wash thoroughly after handling. Wear protective gloves/clothing/eye protection/face protection. Absorb spillage to prevent material damage. Store in a cool, dry, well-ventilated area away from heat, direct sunlight and all sources of ignition. Store away from incompatible materials. Keep container closed to prevent contamination. Dispose of contents and container in accordance with local and national regulations. **FIRST AID:** **IF ON SKIN (or hair):** Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. **IF SWALLOWED:** Rinse mouth. Do NOT induce vomiting. **IF INHALED:** Remove victim to fresh air and keep at rest in a position comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. **FIRE:** Not combustible. Use extinguishing media suitable for surrounding. Under fire conditions, toxic, corrosive fumes are emitted.

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