# primobyte

**lichthärtendes Bissregistratmaterial  
light cured bite registration material**

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## Product description

Developed in cooperation with  
Dr. Diether Reusch / Westerbürg, Germany.

primobyte centric bite registration kit consists of two components, the green centric registration base plate and the blue detail paste (for the actual centric stops).

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## Uses

Light cured primobyte centric bite registrates can be extremely thin and yet very solid to ensure minimal lifting of the vertical dimension when the models are mounted.

Unlike wax, the primobyte centric registrates perfectly maintain their shape and dimension, are not temperature sensitive and strong, so in general just one registrate per case is sufficient.

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## Advantages

**primobyte centric bite registrates consist of two components, the centric registration base plate and the detail paste.**

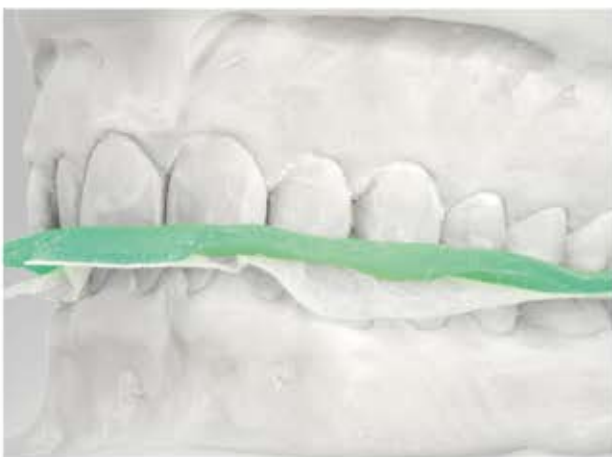
primobyte centric registration base plates ensure perfect passive fit without clinically relevant polymerization-shrinkage, as they were developed especially for bite registration purposes.

They can even be prepared in the laboratory which greatly reduces chair side time.

During the actual registration procedure (before being light cured) primobyte detail paste is very soft and still nicely staying in place during the actual registration procedure. It does not stick to the opposing teeth.

Compared to bite registration waxes (aluminum wax) the dentist can decide when he would like primobyte detail paste to harden (by light curing).

primobyte has no taste or smell, is economical in use, highly efficient and produces great results from the very beginning. It is easy in handling and helps to make centric bite registration really enjoyable.

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*For the preparation of the primobyte centric registration base plate, models of hard dental stone (min. class III material) are required.*

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*The models need to be mounted with habitual intercuspitation in the articulator. An adequate separator (stone/acrylic) has to be applied on the upper model occlusally (i.e. primosep or Metatouch from primotec).*

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*Both sides of the primobyte centric registration base plate are protected by separating paper. One paper needs to be removed on one side of the base plate.*

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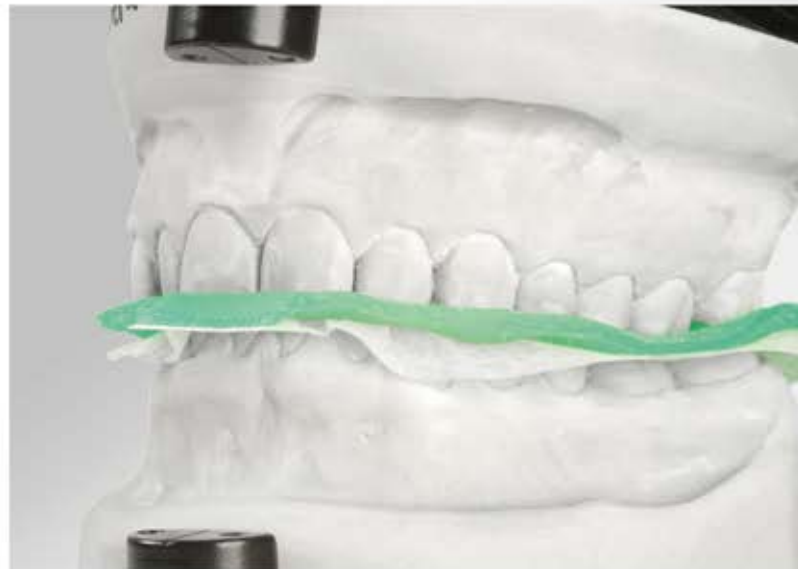
*The centric registration base plate is adapted to the upper model with the remaining separating paper facing towards the lower model. No separator needs to be applied on the lower model.*

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
*After the initial adaptation, the articulator is closed strongly in order to press down and thin out the centric registration base plate to the max. The upper model with the readily adapted centric registration base plate in place, is removed from the articulator.*

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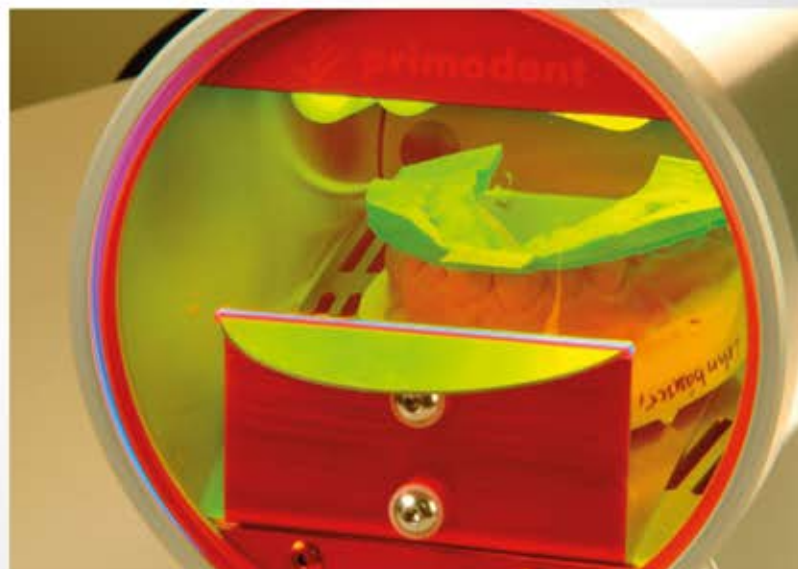*The base plate on the model is polymerized in an adequate light curing unit (Lumos by primodent or similar) for 3-6 minutes. Then the base plate is removed from the model and polymerized for another 3-6 minutes from the reverse side.*

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*After light curing, the polymerized primobyte centric registration base plate has to be finished by grinding with carbide burs (used for acrylic).*

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*First, the base plate is shortened distally of the last two antagonizing teeth and along the connecting line of the upper buccal cusps*

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*Second, sharp edges are rounded off with sandpaper. Finally the registration plate is cleaned and disinfected.*

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*Now the primobite centric registration base plate is ready to be used in the patients mouth.*

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*Materials and tools needed for the centric bite registration:*

- red articulating silk
- Schimstock foil
- pencil
- handpiece with diamond bur
- cotton swab
- primobyte detail paste

# Centric bite registration procedure

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*The centric registration base plate is positioned on the patients upper teeth and the perfect passive fit is verified.*

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*The contact points of the lower canines as well as the first and second molar are marked by utilizing the red articulating silk.*

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*To minimize the vertical lifting, the registration base plate needs to be thinned out as much as possible.*

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*The following lower contacts (in centric condyle position) need to be cross-marked with a pencil:*

- *interproximal area between lower laterals and canines*
- *buccal groove of the lower last molar*

# Centric bite registration procedure

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*primobyte detail paste is applied on the anterior cross-marks.*

**14.**

Patient mehrmals in Zentrik führen.

*The patient needs to be guided into centric position several times.*

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*The centric registration base plate is removed from the patients mouth and the added primobyte detail paste is polymerized in an adequate light curing unit (ideally Lumos by primodent).*

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*Excess primobyte detail paste is removed by grinding with a diamond bur (max. depth of impression should be 0,5 mm).*

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*primobyte detail paste is applied on the posterior cross-marks and the patient is again guided into centric position several times.*

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*The centric registration base plate is removed from the patients mouth and the newly added primobyte detail paste is polymerized in an adequate light curing unit (ideally Lumos by primodent or primotec Metalight).*

# Centric bite registration procedure

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*The perfect fit of the primobyte detail paste has to be finally verified in the patients mouth using Schimstock foil in addition, the perfect fit of the final primobyte centric bite registrate on the lower model has to be checked.  
Finished!*

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**PB101**

## Order information

**PB001** primobyte Start Set, containing 36 centric registration base plates 6 syringes of detail paste, 3g each

**PB101** primobyte centric registration base plates refill. Containing 72 centric registration base plates

**PB201** primobyte detail paste in syringes, containing 10 syringes detail paste 3g each

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primotec  
Joachim Mosch e.K.  
Tannenwaldallee 4  
DE-61348 Bad Homburg

## Notes about primobyte

- **Composition primobyte**  
oligomere acrylates, anorganic and organic fillers, photo initiators.

- **Storage notice**  
**primobyte** registration base plate and detail paste should be stored and used at temperatures between 4°C und 25°C. All materials shall be protected against heat and direct sunlight.

Always close the containers right after use. The expiration date is printed on the product label. After the materials are expired they must not be used anymore.

- **primobyte products warning**  
**primobyte** contains different acrylates, fillers, glass perls, starter for polymerization and admixtures.  
In case of any irritations occur by using the product, or if the user is aware that he might react allergic to one of the products components, the material should not be used.

To avoid any health risks, especially grinding should only be done at a work bench with an appropriate suction system (for further information check the primobyte safety data sheets).

- **Regulations for use**  
**primobyte** products were developed for the use in the dental field. They have to be used in accordance with this instruction manual. For any damages caused by the use of the products in any other than the dental field, neither the manufacturer nor the distributor will take any responsibility. In addition, the user is obliged to verify, at his own responsibility, whether the material can be used for the desired purpose, especially if the desired purpose is not described in the instruction manual.

# INNOVATIONS FOR DENTISTRY

Joachim Mosch e.K.  
Tannenwaldallee 4  
D-61348 Bad Homburg  
Fon +49 (0) 6172 - 99 770 - 50  
Fax +49 (0) 6172 - 99 770 - 99  
[www.primogroup.de](http://www.primogroup.de)  
[primodent@primogroup.de](mailto:primodent@primogroup.de)

Primotec USA  
360 Westport Ave.  
Norwalk, CT 06851  
(866) 643-3129  
[www.primotecusa.com](http://www.primotecusa.com)  
[info@primotecusa.com](mailto:info@primotecusa.com)