MTA ANGELUS

CHOOSE YOUR LANGUAGE

ENGLISH	4	HRVATSKI	33
ESPAÑOL	6	SLOVENŠČINA	36
PORTUGUÊS	9	SLOVENSKY	38
FRANÇAIS	12	POLSKI	41
ITALIANO	14	LIETUVIŠKAI	44
ROMÂNĂ	17	ČESKY	46
NEDERLANDS	20	MAGYAR	49
DEUTSCH	23	TÜRKÇE	52
NORSK.....	25	БЪЛГАРСКИ	54
SVENSKA	28	ΕΛΛΗΝΙΚΑ	57
SUOMI	30		

ENGLISH

DEFINITION

MTA ANGELUS® belongs to the family of endodontic cements, mainly composed of Mineral Trioxide Aggregate (MTA), intended for patients who require endodontic procedures, with the function of repairing endodontic complications in the root canal of the tooth in cases of lateral and furca perforations, internal resorption, root-end filling, pulp capping, pulpotomy, apexification and apexogenesis.

INDICATIONS

1. Treatment of perforations of root canal and furcation caused iatrogenically or by caries lesion (2) (Pics. 1 and 2);
2. Via canal treatment of root perforation due to internal resorption (2) (Pic. 3);
3. Surgical treatment of root perforation due to internal resorption (2) (Pic. 4);
4. Periapical surgery with reverse filling (5) (Pic. 5);
5. Direct pulp capping (7);
6. Pulpotomy (removal of affected coronal pulp to preserve vitality of remaining pulp tissue) (2) (Pic. 6);
7. Apexogenesis (induction of root development in vital teeth with an inflamed coronal pulp) (2);
8. Apexification (induction of formation of a mineralized barrier at the root tip of young permanent teeth with incomplete root development and a necrotic pulp) (2) (Pic. 7).

INTENDED USER

MTA ANGELUS® is intended for use by dentists who are trained to perform endodontic procedures.

CONTRAINDICATIONS

MTA ANGELUS® contains components that may cause reactions in susceptible individuals. Do not use on patients who are allergic to the product's components.

COMPOSITION

- MTA ANGELUS® Gray: Tricalcium silicate, dicalcium silicate, tricalcium aluminate, calcium oxide, iron tetracalcium aluminate, bismuth oxide;
- MTA ANGELUS® White: Tricalcium silicate, dicalcium silicate, tricalcium aluminate, calcium oxide, calcium tungstate.

TECHNICAL INFORMATION

- Hydrophilic: can be used in humid areas without losing properties;
- Adequate radiopacity: allowing radiographic visualization;

- Low solubility: does not allow leakage;
- High alkalinity: inhibits bacterial growth;
- Setting time of 15 minutes: less than similar cements;
- Assists in the formation of mineralized tissues inducing the neoformation of periradicular cementum, providing sealing of perforations and repair of damaged periradicular tissues;
- MTA ANGELUS® WHITE: does not stain the dentinary structure (without bismuth oxide).

WARNINGS

- Avoid contact with eyes or skin. In case of contact, rinse immediately with water;
- Avoid contact with oral mucosa. In case of contact, rinse with water and avoid ingesting the product. If any sensitivity persists, seek medical attention immediately.

PRECAUTIONS

- Use protective glasses, masks, and gloves when handling MTA ANGELUS®. In case of contact with the product with eyes or skin, wash with water;
- Only open the sachet or bottle immediately before use. MTA ANGELUS® is very sensitive to moisture. Tightly close the bottle after each use;
- Use the product only after controlling the acute phase of the endodontic disease. The acidic pH of endodontic lesions and their surrounding areas can compromise the solidification of MTA ANGELUS®;
- Apply MTA ANGELUS® carefully. Like other cements, it is normally reabsorbed, but any excess may make healing difficult.
- Do not use MTA ANGELUS® to fill root canals, because its plasticity and drainage are inadequate for this purpose and, in case of retreatment, its removal would be difficult;
- Do not use MTA ANGELUS® in places that are in contact with the gingival groove, because a complete solubilization of the cement will occur;
- Do not use MTA ANGELUS® GRAY in areas exposed to light to ensure that staining of the dental structure does not occur;
- Do not reuse any spare powder or liquid from the glass plate;
- The safety and effectiveness of use have not been established for pregnant or breastfeeding women or for children or adolescents.

ATTENTION: The success of the procedures with MTA ANGELUS® depends on a correct diagnosis, a precise operative technique, the conditions of the tooth under treatment and the patient's systemic condition. This product must be used according to the instructions in this package insert.

TECHNIQUE OF USE

1. Sterilize a glass slab, a metal spatula and all instruments for the insertion and condensation of MTA ANGELUS®;
2. Mix for 30 seconds the content of 1 sachet of MTA ANGELUS® (or 1 spoon of MTA ANGELUS®) with 1 drop of distilled water. The mixture should be homogeneous and with a consistency similar to wet sand;
3. Apply the MTA to the desired location with an ANGELUS MTA APPLICATOR or a suitable instrument;
4. Condense the cement with instruments such as amalgam condensers, a number 1 spatula or absorbent paper points moistened with distilled water;
5. Take a radiograph for control.

IMPORTANT: If MTA ANGELUS® is not used immediately after mixing, its dehydration can be prevented and the working time increased by covering the mix on the glass slab with a wet gauze. If the mixed MTA dehydrates, it must be discarded.

STORAGE

Keep in a dry and well ventilated place and away from light.

DISPOSAL METHOD

The product must be disposed of in accordance with local regulations.

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