Instructions for use

- EN** see www.fkg.ch for additional information
- FR** informations supplémentaires sur www.fkg.ch
- DE** weitere Informationen auf www.fkg.ch

XP-endo® Finisher

Endodontic instruments
Instruments endodontiques
Endodontische Instrumente

FKG REF. 99.X00.10.01A.XX - n°108 - 2021/12

XP-endo® Finisher

Description - Beschreibung

25/00

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Golden rules

Speed : 800-1000 rpm
Torque : 1 Ncm

- XP-endo® Finisher should be used only after canal preparation to at least ISO 25.
- In multirrooted teeth, begin with the largest canal.
- Work along the entire length of the canal for approximately 1 minute.

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General information

Indications for use :

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

The XP-endo® Finisher is used as a final cleaning instrument in root canal treatments.

Contraindications :

This product contains nickel, and must not be used on individuals with a known allergic sensitivity to this metal.

Adverse reactions : None known

Warnings and precautions :

• Single use instruments must not be retreated for subsequent use. That could compromise their integrity.

• Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).

• When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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This is to certify that:

Emergo Global Consulting, LLC

2500 Bee Cave Road
Building 1, Suite 300
Austin
Texas
78746
USA

Holds Certificate Number:

FS 594789

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 for the following scope:

Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry.

For and on behalf of BSI:

Graeme Tunbridge, Senior Vice President Medical Devices

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 1 of 2

...making excellence a habit.™

Certificate No: FS 594789

Location	Registered Activities
Emergo Global Consulting, LLC 2500 Bee Cave Road Building 1, Suite 300 Austin Texas 78746 USA	Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry. AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations.
I.L. Emergo Israel Ltd. Andrei Sakharov 9 Building 25 - 8th floor Matam North Haifa 3190501 Israel	AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations.
Emergo Europe Westervoortsedijk 60 6827 AT, Arnhem The Netherlands	Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation delivered to the medical device industry.

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 2 of 2

This certificate was issued electronically and remains the property of BSI and is bound by the conditions of contract.

An electronic certificate can be authenticated [online](#).

Printed copies can be validated at www.bsigroup.com/ClientDirectory

Issuing Body: BSI Group The Netherlands B.V., John M. Keynesplein 9, 1066 EP Amsterdam, The Netherlands

BSI Group The Netherlands B.V. is registered in The Netherlands under number 33264284 | A Member of the BSI Group Holdings B.V.

Contact Office: 12950 Worldgate Drive, Suite 800, Herndon, VA 20170-6007 USA.