XP-endo® Rise Retreatment

1 blister includes / contient / enthält: DR1, XP-endo® Rise Shaper 30/.04, Finisher R 30/.00

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General information

Indications for use:

The endodontic XP-endo® Rise Retreatment instruments are intended for use in medical or hospital facilities, by qualified health professionals. This sequence is used for endodontic retreatments.

Contraindications:

This product contains nickel and must not be used on individuals with a known allergic sensitivity to this metal.

Adverse reactions: None known.

Notice and precautions:

- Take several radiographs from different angles to determine the anatomy of the root canals (length, width and curvature).
- Single use instruments must not be retreated for subsequent use. That could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- Use the instruments XP-endo® Rise Retreatment in the order of the sequence.
- Irrigate the canal thoroughly and frequently after the XP-endo® Rise Retreatment instrument is extracted.
- Regularly inspect the XP-endo® Rise Retreatment instrument during use, and discard if it shows any signs of wear (for example: straightening).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.
- These instruments have not been tested on children, pregnant nor breastfeeding women.
- Visit the FKG Dentaire SA website (www.fkg.ch) for all information relating to the applicable cleaning and sterilization methods.

Protocol for use

1. Gently engage DR1 rotating at 800-1000 rpm and 1.5 Ncm torque into the obturation material in order to create a 3-4 mm starting point for the XP-endo® Rise Shaper tip (Fig. 1). The active tip of DR1 facilitates the initial penetration.

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

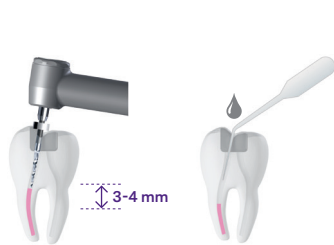
XP-endo® Rise Shaper

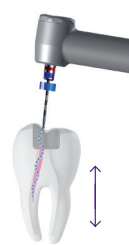Finisher R

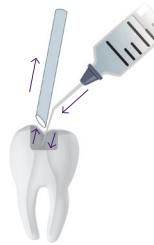- Place one drop of solvent in the coronal space created and wait for at least 1 minute (Fig. 2).
- Place the tip of XP-endo® Rise Shaper into the prepared space in the gutta percha, disengage slightly and start the motor at 1000-2500 rpm (Fig. 3). *Recommended: 1000 rpm - 1 Ncm.*
- Use pecking motion until the XP-endo® Rise Shaper's tip engages the gutta percha. Then use light pressure to help advance the XP-endo® Rise Shaper down the canal until working length or the end of the previous filling material is reached (Fig. 4). Add solvent if needed.
- Allow the XP-endo® Rise Shaper to gently corkscrew around the gutta percha and tease out large strands of gutta percha using long gentle strokes (Fig. 5). Note: you may need to remove gutta percha from the surface of the instrument.
- After the removal of the gutta percha, determine the working length (WL) with an electronic apex locator.
- Once the WL is confirmed, achieve this length with the XP-endo® Rise Shaper, irrigate and fill the pulp chamber and canals with NaOCl.
- Use the XP-endo® Rise Shaper for 10-15 additional long gentle strokes to WL and irrigate the canal to eliminate suspended debris.
- Fix the canal working length on the Finisher R (ISO 30) by using the plastic tube to adjust the endo stop.
- Place the Finisher R into the canal, then start the motor (800-1000 rpm, 1 Ncm) and slowly thread the Finisher R into the canal.
- Use the Finisher R for at least 1 minute (longer, if debris is still coming out of the canal), using slow and gentle longitudinal movements to contact the full length of the canal. Make small multidirectional movements and avoid removing the instrument from the canal (Fig. 6).
- Use copious NaOCl irrigation and suction to remove the gutta percha and sealer tags prior to complete the treatment (Fig. 7).

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