LYNX

QUICK GUIDE

**Congratulations
on your new LYNX.**

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**Keep the original packaging;
in the case of a return or
maintenance, it will be
necessary to send the product
to your authorised dealer.**

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In this quick guide you will find all the useful information for the correct use and maintenance of your LYNX.

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Check this link to access information in languages not found in this guide.

Index

- 01. **Components** / *Componenti • Komponenten • Composants • Componentes*
- 02. **Control unit recharge** / *Ricarica unità di controllo • Aufladen des Steuergeräts • Carga de la unidad de control • Recharge de l'unité de contrôle*
- 03. **Power ON/OFF** / *Accensione e spegnimento • Ein- und Ausschalten • Encendido y apagado • Mise en marche et arrêt*
- 04. **Light intensity** / *Intensità luminosa • Lichtstärke • Intensidad luminosa • Intensité lumineuse*
- 05. **Assembly & adjustments** / *Montaggi e regolazioni • Montage und Einstellungen • Montaje y ajustes • Montages et réglages*
- 06. **Optional** / *Optional • Optional • Opcional • Options*
- 07. **Instruction for use** / *Nota informativa • Gebrauchsanweisung • Instrucciones de uso • Mode d'emploi*

Components

Componenti

Komponenten

Composants

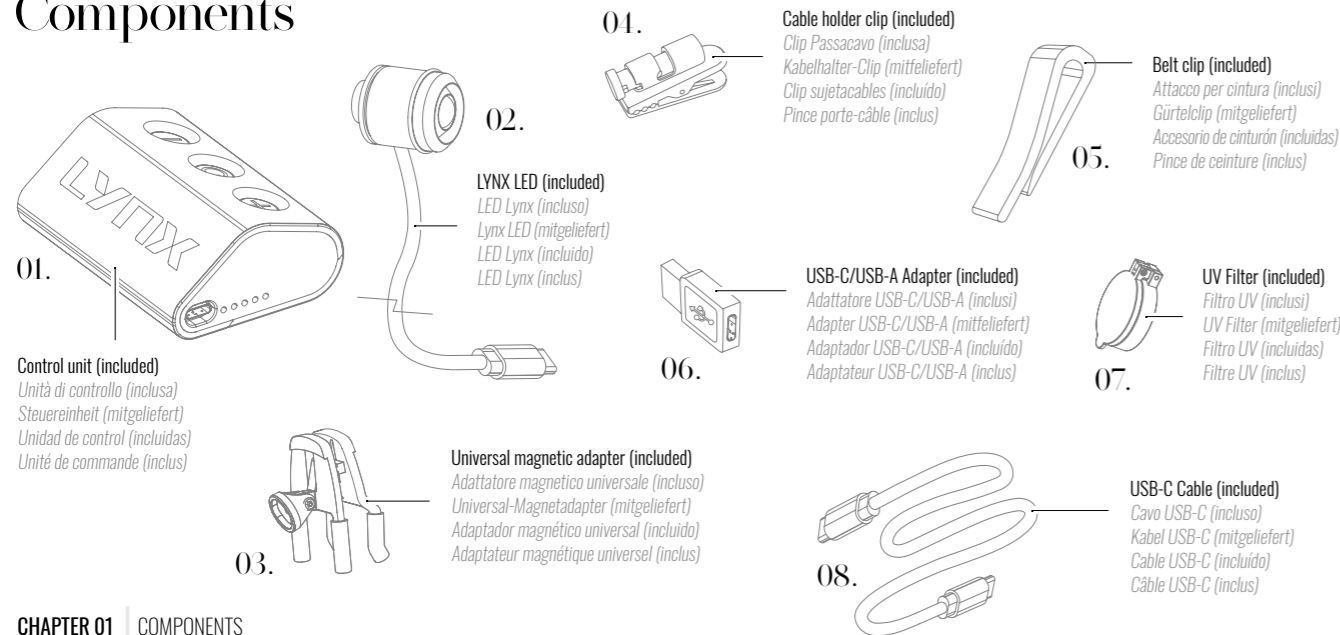
Componentes

Chapter


Components

7

Components

Where to find the components

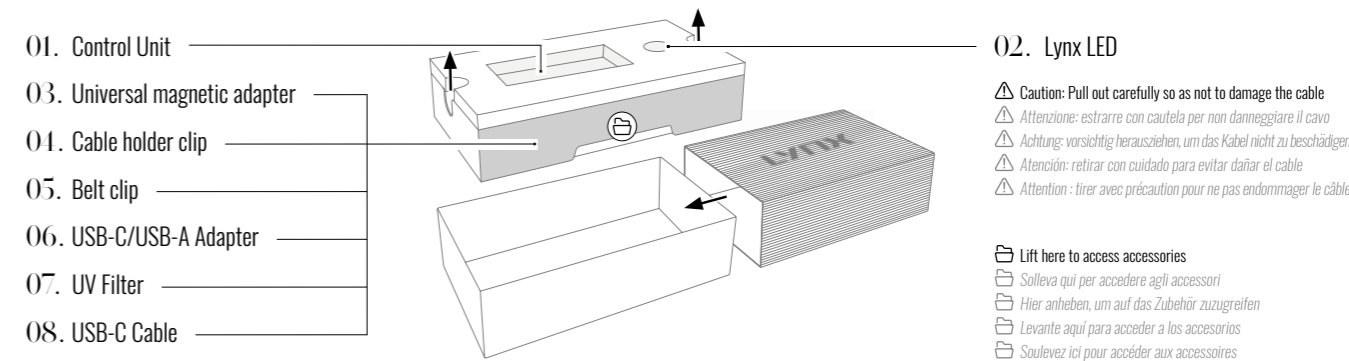CAUTION

Follow all steps in the sequence indicated for the optimum correct result.

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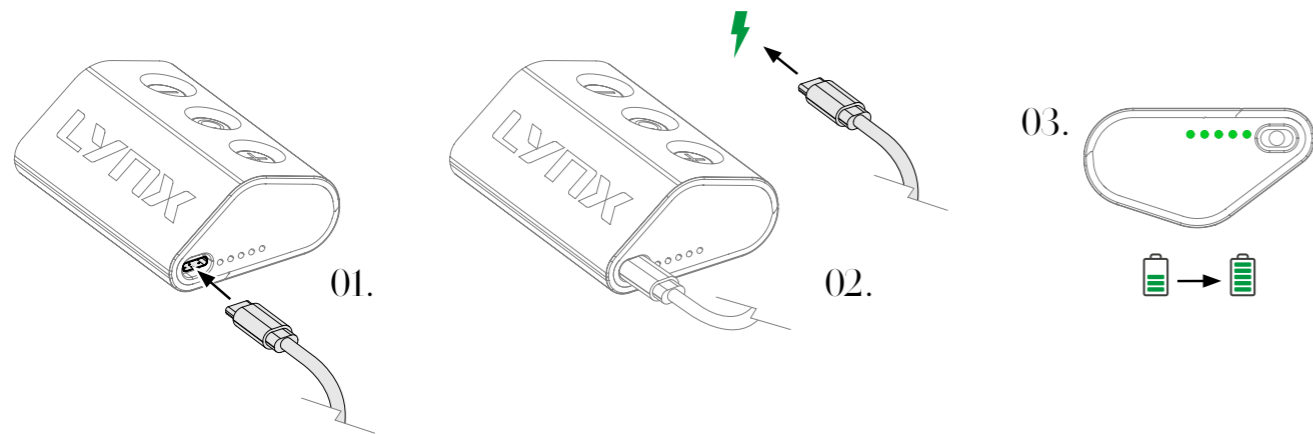
Control unit recharge

Control unit recharge

To recharge the control unit, it will be necessary to connect one end of the cable "USB - type C" to the control unit (Fig. 01) and the other end to a power source (Fig. 02). The charging action of the device is signaled by the green LED indicators lighting up. The device will be fully charged when all LED indicators are lit solid green (Fig. 03).

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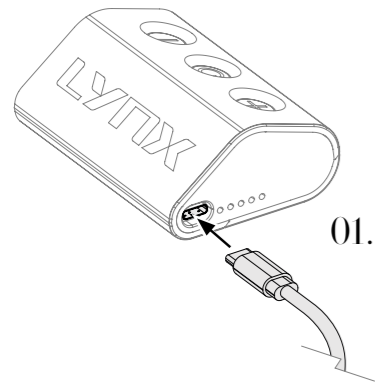
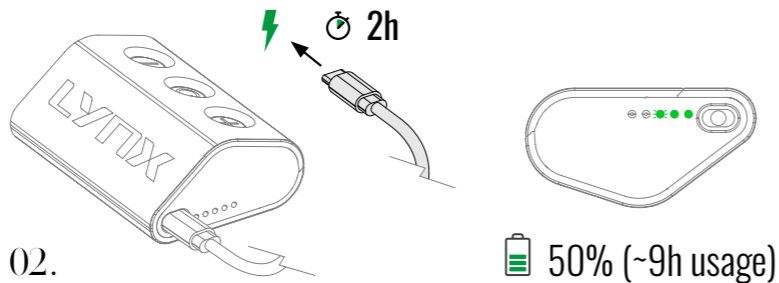
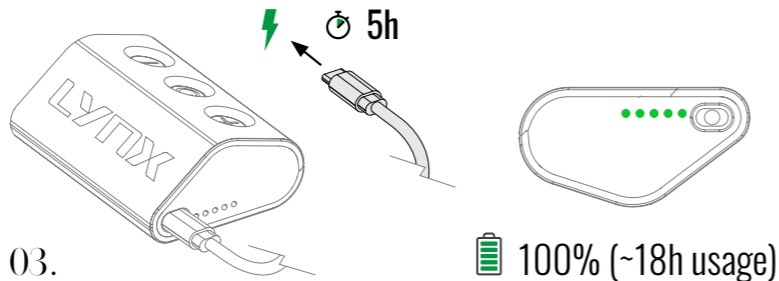
03.

Control unit charging time

If the device is fully discharged, it can be recharged (Fig.01):

- At 50% in about 2h (about ~9h working time at full luminosity) (Fig.02).
- At 100% in about 5h (about 18h working time at full luminosity) (Fig.03).

The charging times mentioned are valid when using the recommended power source (info at pag. 57)

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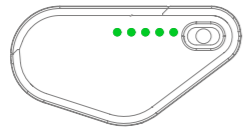
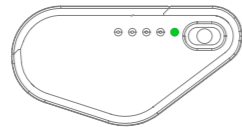
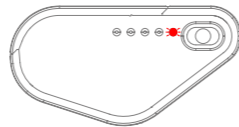
 < 5%

Battery charge levels

The led indicators on the control unit will show green if the device is charged, according to 5 charge levels ranging from 100% to 5% (Fig.01). Below 5% charge, the first led indicator (Fig.02) will flash red, signaling the imminent shutdown of the device.

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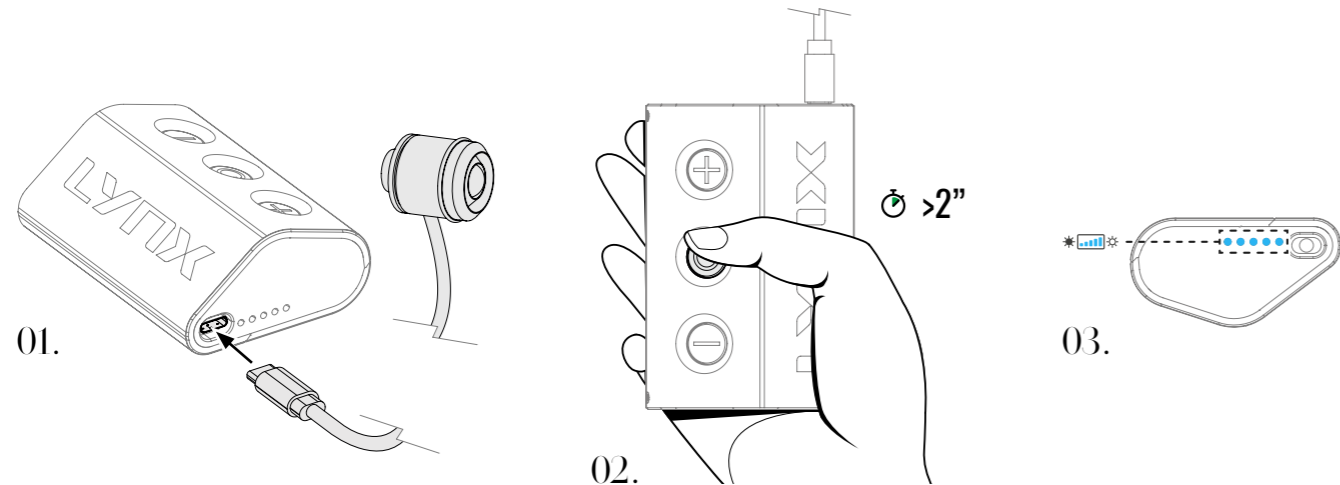
Turning the device ON

Turning on the device

Connect the LED spotlight to the control unit by plugging the spotlight's USB-C cable into the appropriate connector (Fig. 01). Then press and hold the ON/OFF button for at least 2 seconds to turn on the spotlight (Fig. 02). The LED indicators on the control unit will light blue to indicate the brightness level of the LED (Fig. 03).

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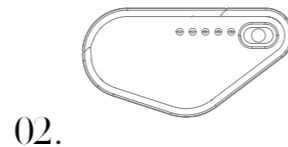
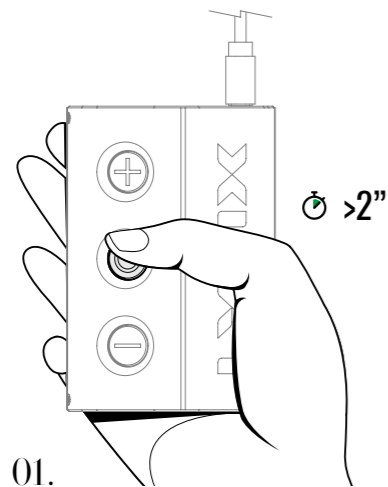
Turning the device OFF

Switching off the device

Press and hold the ON/OFF button for at least 2 seconds to turn off the illuminator (Fig. 01). All led indicators on the control unit will turn off (Fig.02).

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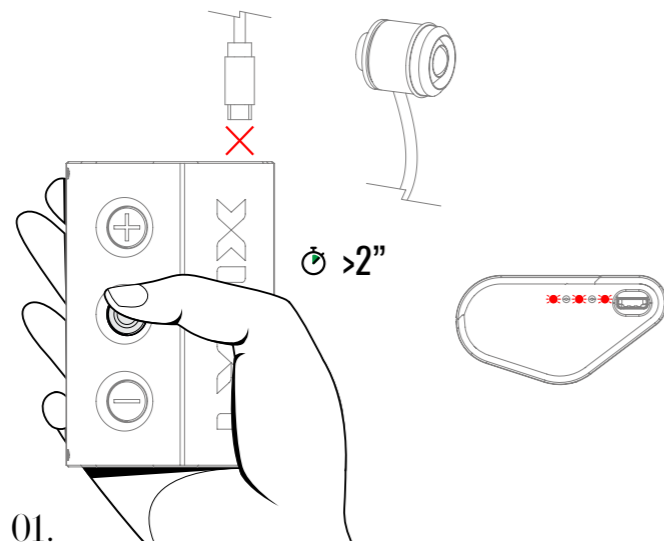
CAUTION: Error signal

When turning on the device, the error will be indicated by 3 red LED indicators (Fig. 01) in case:

- The control unit was disconnected from the LED or power supply;
- The 'control unit was connected to the LED incorrectly or an abnormality occurred;
- The control unit was connected to the power supply incorrectly or an abnormality occurred.

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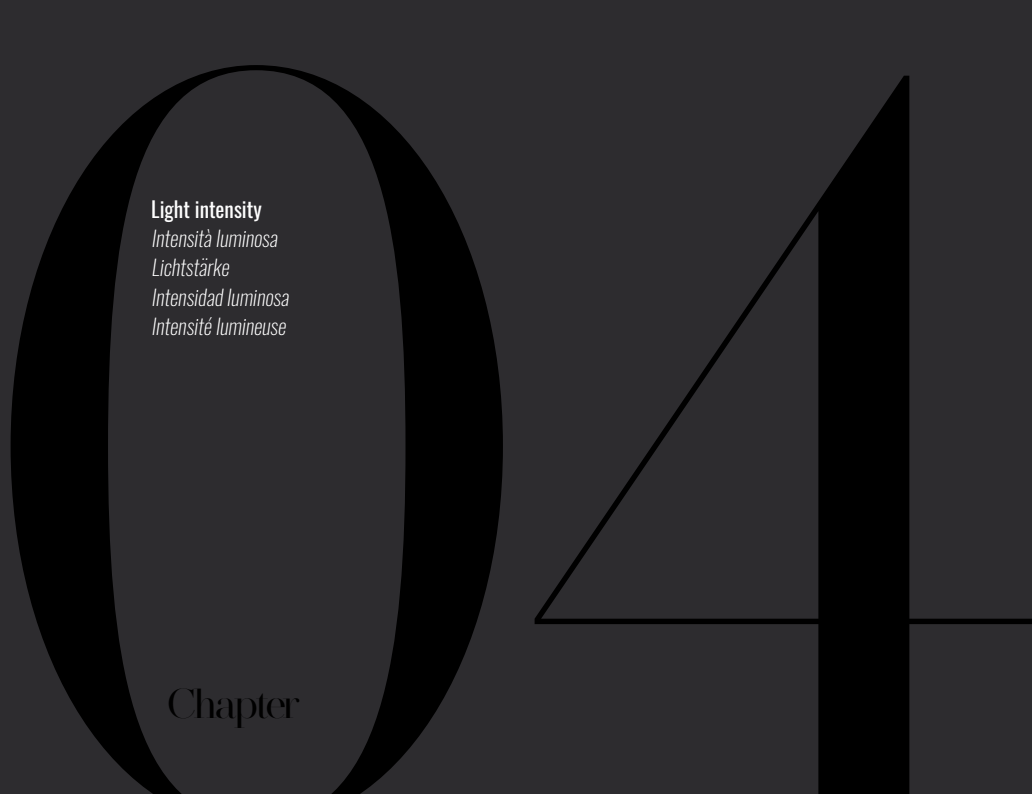
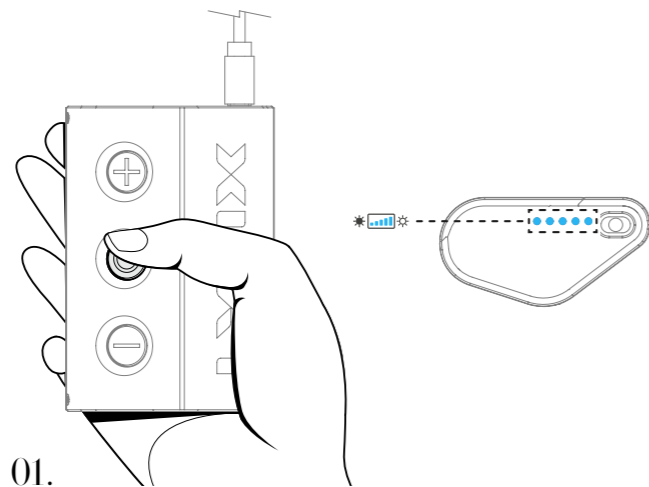
Light intensity levels

Light intensity level indication

Once the Headlight is turned on, the LED indicators on the control unit (Fig. 01) will light blue, signaling the level of LED light intensity.

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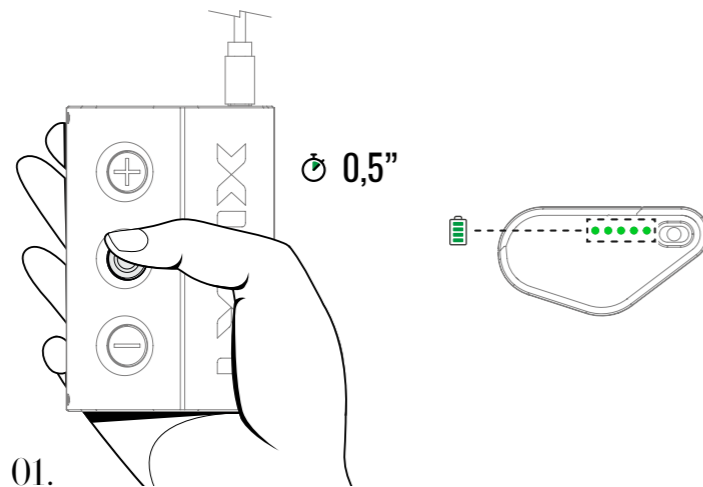
Switch function

Switching from luminous Intensity to Charge Status

With the device switched on, it is possible to switch from the indication of the luminous intensity to that of the battery charge status by quickly pressing the ON/OFF button (Fig. 01).

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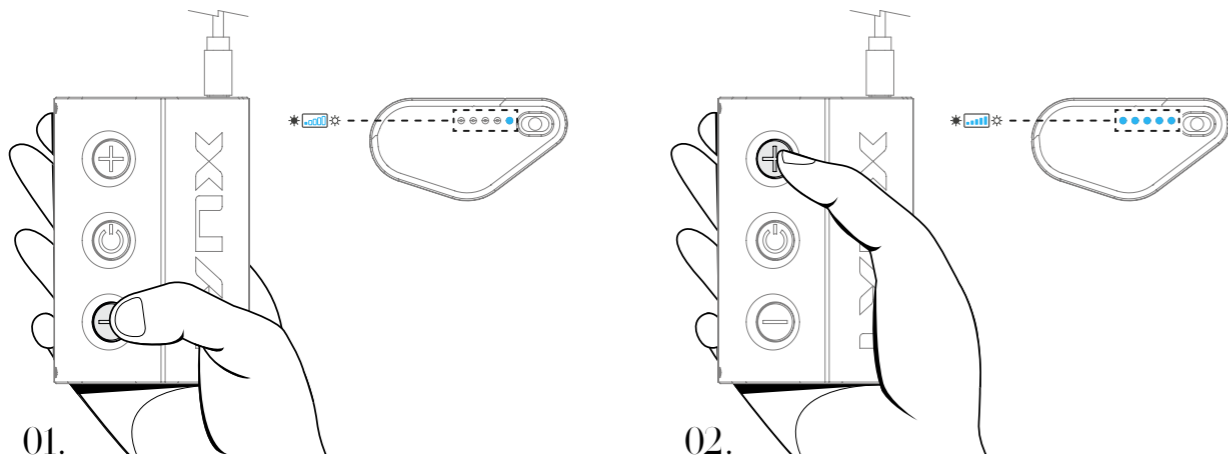
Legend of the light intensity levels

Light intensity levels

The light emitted by the LED can be adjusted according to 5 levels of intensity using the minus (-) and plus (+) buttons. Each time the minus (-) key is clicked, the light intensity level will gradually decrease to a minimum of 1 (Fig. 01). On the contrary, with each click of the plus (+) key, the brightness level will gradually increase up to a maximum of 5 (Fig. 02).

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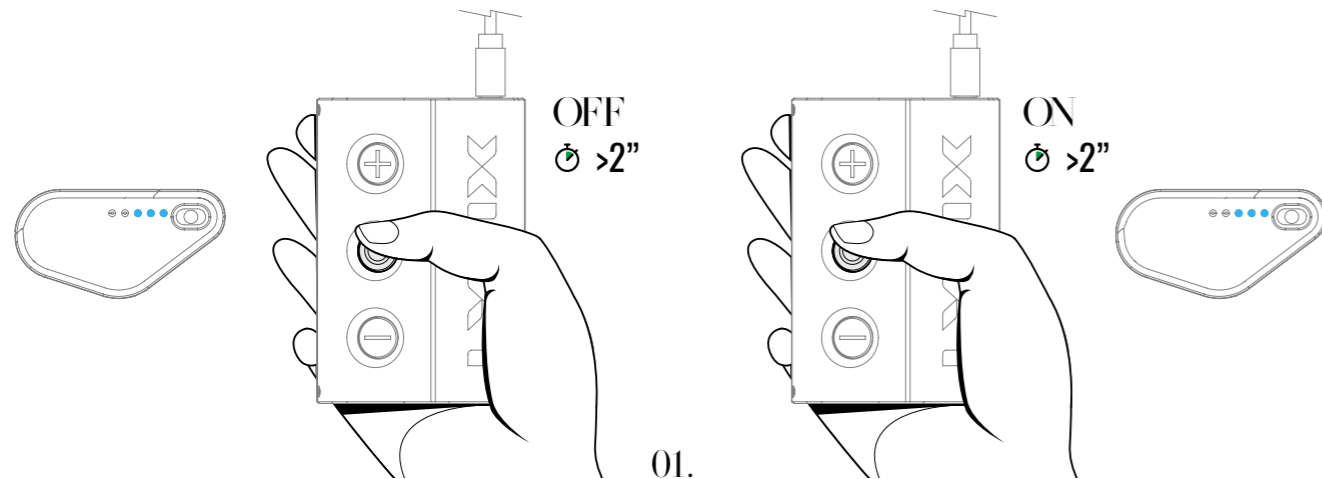
Saving light intensity

01.

Light intensity level memorization

The brightness level selected before switching off the device will be memorised and re-proposed when turned on again (Fig. 01).

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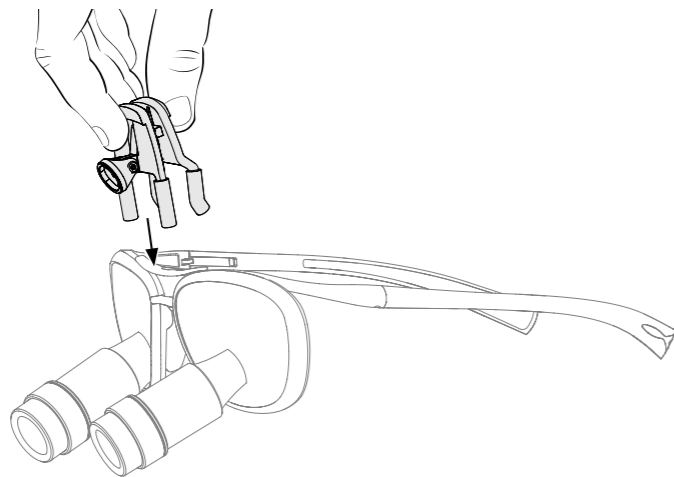
Assembly & Adjustments

37

Attachment to the magnifying system

Hook with universal adapter

Open the clip of the adapter (included in the package) and place it over the nose bridge of the loupes.

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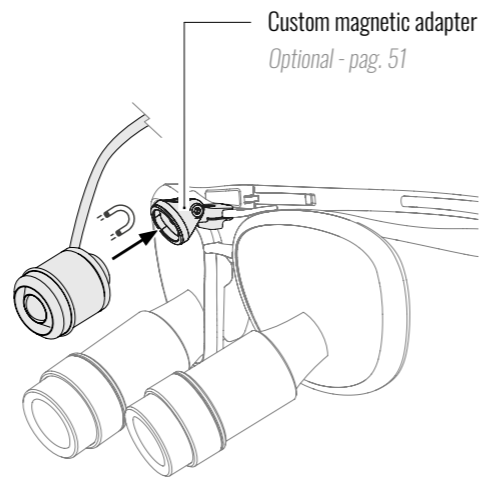
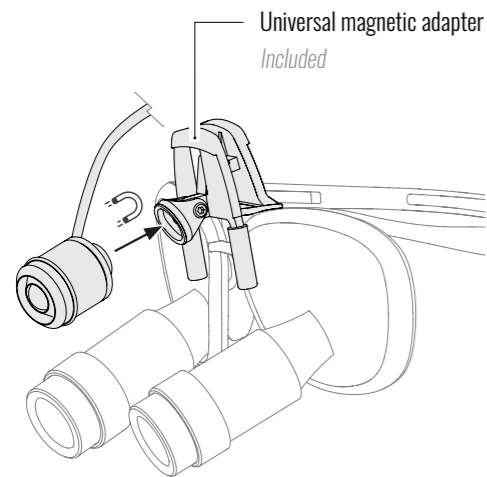
Attachment to the magnifying system

Led headlight fastening to the universal or custom adapter

After attaching the adapter to the loupes, attach the LED headlight to the adapter using the magnet. The operation can be performed in the same way on both the universal and the custom adapter.

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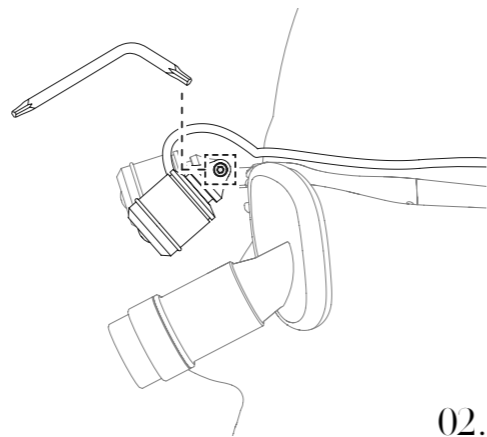
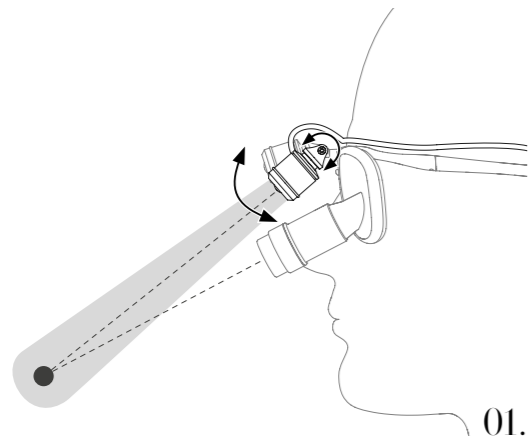
Led adjustment

Adjusting the LED

We advise you to adjust the inclination of the LED headlight so that the light spot illuminates the field of view, acting on the joint of the magnetic coupling and on the joint of the LED headlight. Once the correct position has been established (Fig.01), we suggest you tighten the screws (Fig.02) using the Allen key (included in the package).

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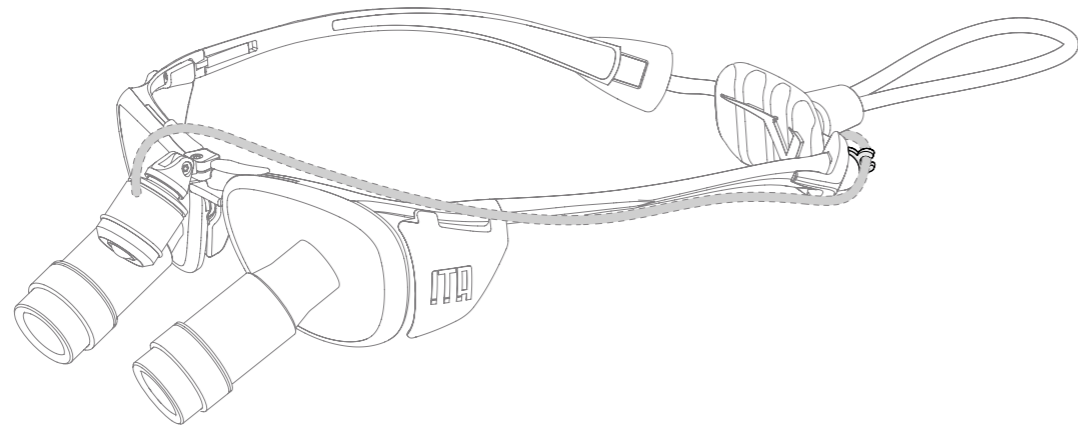
Use of the cable gland

To avoid hindrances caused by the LED cable during its use, we recommend you fasten it through the cable glands to the frame of your loupes (for more information download the quick guide dedicated to your Loupes at the page www.univetloupes.com/userguide). In addition, you can also use the cable holder clip, included in the product, to avoid cable movement during operation.

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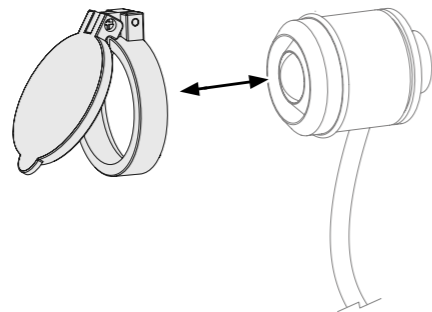
Additional assembly

UV filter fastening

With a light pressure fasten the UV filter (included in the package) on the front of the LED headlight.

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A large, stylized number '6' is rendered in a dark grey color against a black background. The number is composed of thick, curved strokes. The top part of the '6' is a large, rounded shape that tapers to a point on the right. The bottom part is a smaller, rounded shape that also tapers to a point on the right, creating a continuous, flowing line.

Optional

Optional

Optional

Opcional

Options

Chapter

Optional

49

Custom magnetic adapter

Cod. KZ41718.OA

ITA Loupes

Custom Headlight Adapter

Attacco illuminatore

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Cod. KZA005531

ONE/LOOK&COOL Loupes

Custom Headlight Adapter

Attacco illuminatore

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Cod. KZ25818.OA

TECHNE/TECHNE RX Loupes

Custom Headlight Adapter

Attacco illuminatore

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Cod. KZ25918.OA

ASH Loupes

Custom Headlight Adapter

Attacco illuminatore

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Optional

Our custom, model-specific magnetic adapters allow you to quickly and comfortably mount your headlight to your loupes, transforming loupes and light into an integrated system.

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For more information or inquiries regarding replacement parts, please contact your trusted Univet Authorized Dealer, or contact us directly by scanning the QR code or writing to us at:
info@univetloupes.com

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Instruction for use

55

**Instructions for use - Lynx LIGHTING SYSTEM**

Thank you for your trust in us and congratulations on your choice.
Read the following information before using the device.

DESCRIPTION OF THE COMPONENTS

The list of components and spare parts are indicated in the documents provided along with the device.

SYMBOLS

Below is the symbols present on the device label (when applicable):

In accordance with Regulation (EU) 2017/745 on medical devices

This product cannot be dissolved in unsorted municipal waste

Country of manufacture
Date of manufacture

Manufacturer

Instructions for use

Medical Device

Batch number

Catalogue number

Serial number

Caution

Humidity limitation

Atmospheric pressure limitation

Temperature limitation

INTENDED USE

The Univet Lynx LED (Light Emitting Diode) system is a portable lighting device that, when worn by the operator, illuminates the visual field during work activities.

The lighting system is an accessory intended to be used, in combination with a "magnifying system" medical device, to illuminate the area viewed by the user.

Intended users: medical professionals.

Intended patients: no limitation except as stated in the "contraindications" section.

DEVICES THAT CAN BE USED TOGETHER

The device can be comfortably attached to all Univet magnifying system models, thus providing an integrated vision system.

CONTRAINDICATIONS

The device is contraindicated in the event of:

- Patients highly sensitive to light;
- Applications requiring lighting stability that cannot be achieved with devices mounted on the user's head.

WARNINGS

Read the following warnings carefully before using the device.

Failure to follow these safety instructions could result in fire, electric shock, injury or damage to the LED lighting system or to other property. Do not use the device if it has been damaged. Contact your authorised dealer. Do not tamper with or disassemble any component of the system. Do not attempt any repairs unless expressly indicated. The Lynx device has a built-in rechargeable lithium battery, which cannot be replaced or disassembled by the user or by unauthorized personnel. The battery must be replaced and disassembled exclusively by personnel authorized by Univet. Do not try to change the battery by yourself: improper substitution or the use of wrong spare part can cause risk of explosion. Do not expose the device to conditions that could ignite the battery. Avoid contact with heat sources or open flames. Do not use in spaces with flammable anesthetic mixtures. Do not immerse any component in liquid substances. Prevent liquid substances from entering the slots or the connection port. Do not look directly at the light emitted by the LED or direct the light to the patient's eyes. Use the supplied components only: do not combine the components of the system with other lighting systems or electronic equipment. Use only the power supply if supplied or compliant with technical specifications indicated in this information note. Avoid sudden movements with your head when using the device. Avoid walking with the device worn. In the event of a malfunction, stop using the device. If mounted correctly, the device must not come into contact with the user's skin. Clean the device before use. When charging with adaptor, the socket-outlet shall be installed near the equipment and shall be easily accessible as the power supply plug allows you to disconnect the device from the mains supply.

PRECAUTIONS

Before operating the device it is recommended to fully charge it, waiting for all green LEDs on the Control Unit to light indicating that it is fully charged. When not in use, store the device in its container. Pay attention to your posture during prolonged use of the device in combination with Univet loupes.

ADVERSE EFFECTS

No adverse effects detected.

SUPPLY MODE

This device is not supplied sterile. This is not a single use device.

STORAGE AND/OR HANDLING

Store the device at the temperatures and humidity indicated in the "Technical Specifications" section. When not in use, store the cleaned device in its original packaging, disconnecting the Control Unit from LED.

MAINTENANCE

Clean the device as described in the "cleaning instructions" section. Keep the original packaging required for return/maintenance shipment and contact your authorised dealer. Any eventual activity / maintenance operation of the device must be carried out only and exclusively by Univet authorized personnel. The device does not need preventive maintenance activities.

TRAINING AND QUALIFICATIONS FOR USE

None.

CLEANING INSTRUCTIONS

Disconnect the LED from all components of the system (magnifying system, Control Unit) before cleaning. Use a microfiber cloth to clean the lens. The LED can be cleaned with a soft cloth soaked with a gentle detergent solution. Do not immerse or spray any components into or with detergents and other liquids. This device cannot be sterilised. Make sure the device is completely dry before use

**DISPOSAL INSTRUCTIONS**

In the event of disposal/demolishing, do not dispose of the device in the environment.
Comply with the relevant national regulations on electronic devices.

HEALTH RISKS

No health risk detected.

WARRANTY

See the instructions laid down by the warranty.
Report any serious accidents involving the device to the manufacturer and the competent authority of the Member state in which the user and/or the patient resides.

TECHNICAL SPECIFICATIONS**Recommended Power supply**

Power supply approved in accordance with IEC 60950-1 standard (SELV approved) or in accordance with IEC IEC 62368-1 standard (ES1 approved) or in accordance with IEC 60601-1 standard (MOOP approved).

Input power supply 100-240 V-; 50-60 Hz; 350 mA;

Output Power Supply 5 V --- 2 A

Control Unit

Battery type Li-Ion / 3,7 V --- 7000 mAh

Charging time > 5 h

Run time approx. 18h ± 10%

LED

Colour temperature 5700 K

Maximum brightness 38000 lux

Risk Group 2

CAUTION: Possibly hazardous optical radiation is emitted from this product. Do not stare at operating lamp. May be harmful to the eyes.

Product tested against IEC 62471

Operating conditions

Temperature da +10°C a +30°C

Relative humidity 10% a 90%;

Atmospheric pressure da 50 kPa a 106 kPa

Storage and transportation condition

Temperature da -10°C a +50°C

Relative humidity 10% a 90%

Atmospheric pressure da 50 kPa a 106 kPa

REGULATIONS / DIRECTIVES / STANDARDS

Compliance: IEC EN 60601-1; IEC EN 60601-1-2; IEC EN 62471.

Unless otherwise indicated, the standards are to be considered in the latest applicable revisions.

Manufacturer's recommendations and declarations Electromagnetic emission		
Emissions test	Compliance	Electromagnetic environment - Guideline
RF emissions according to CISPR 11 EN 55011	Gruppo 1	This device uses RF energy only for its internal operation. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions according to CISPR 11	Class B	The device is suitable for use in a Professional healthcare environment.
Harmonic emissions IEC 61000-3-2	Not applicable	The device is solely battery powered.
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

Further documentation on electromagnetic compatibility (EMC) is available at the manufacturer web site.

For languages not included in these use instructions, please see the following link: www.univetloupes.com/instructions

Univet Loupes Spa – Via G.Prati 87, 25086, Rezzato (BS) Italy - www.univetloupes.com

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Remote Control

UNIVET CONNECT APP

Through the Univet Connect app, you can remotely control your Lynx by using an iOS or Android smartphone. Scan the QR Code and discover all the functions.

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FIND OUT MORE

All specifications and descriptions are subject to change without notice. All rights reserved. Text, images, graphics are subject to copyright and other protective laws. The content of this publication may not be copied, distributed or modified for commercial purposes without written authorization by Univet Loupes Spa.

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P R O

QUICK GUIDE

**Congratulations
on your new LYNX Pro.**

**Register your product now
to activate the warranty and
access content reserved
exclusively for the Univet Loupes
community.**

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For more information or inquiries regarding replacement parts, please contact your trusted Univet Authorized Dealer, or contact us directly by scanning the QR code or writing to us at:

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Instruction for use

55

**Instructions for use - LynxPro LIGHTING SYSTEM**

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Temperature limitation

INTENDED USE

The Univet LynxPro LED (Light Emitting Diode) system is a portable lighting device that, when worn by the operator, illuminates the visual field during work activities.

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Failure to follow these safety instructions could result in fire, electric shock, injury or damage to the LED lighting system or to other property. Do not use the device if it has been damaged. Contact your authorised dealer. Do not tamper with or disassemble any component of the system. Do not attempt any repairs unless expressly indicated. The LynxPro device has a built-in rechargeable lithium battery, which cannot be replaced or disassembled by the user or by unauthorized personnel. The battery must be replaced and disassembled exclusively by personnel authorized by Univet. Do not try to change the battery by yourself: improper substitution or the use of wrong spare part can cause risk of explosion. Do not expose the device to conditions that could ignite the battery. Avoid contact with heat sources or open flames. Do not use in spaces with flammable anesthetic mixtures. Do not immerse any component in liquid substances. Prevent liquid substances from entering the slots or the connection port. Do not look directly at the light emitted by the LED or direct the light to the patient's eyes. Use the supplied components only: do not combine the components of the system with other lighting systems or electronic equipment. Use only the power supply if supplied or compliant with technical specifications indicated in this information note. Avoid sudden movements with your head when using the device. Avoid walking with the device worn. In the event of a malfunction, stop using the device. If mounted correctly, the device must not come into contact with the user's skin. Clean the device before use. When charging with adaptor, the socket-outlet shall be installed near the equipment and shall be easily accessible as the power supply plug allows you to disconnect the device from the mains supply.

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No health risk detected.

WARRANTY

See the instructions laid down by the warranty.

Report any serious accidents involving the device to the manufacturer and the competent authority of the Member state in which the user and/or the patient resides.

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