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Bone and collagen matrix OsteoBiol® - English version -

Class III medical device, bone and collagen matrix of heterologous origin, for the replacement of the autologous bone represented by the matrix and the porous form.

GTO®: heterologous collagenated bone matrix granules, pre-hydrated and mixed with collagen and copolymer gel, mouldable, osteoconductive, resorbable, biocompatible and slightly radioopaque. PARTICLE SIZE: 0,6-1 mm.

CLINICAL INDICATIONS: filler for non-infected, non-sclerotic, vascularized bone defects, on non-load bearing sites.

PACKAGING: ready-to-use, sterile and disposable syringe.

DIRECTIONS FOR USE: After opening the package, the product can be grafted directly into the receiving site, which must be closed by repositioning the flap, followed by suturing. The graft site should be well vascularized and the product well stabilized and protected. We recommend you apply a collagen membrane OsteoBiol® Evolution to protect the graft. In oral surgery, the product must be placed in the superobstetral position.

SIDE EFFECTS: none known. Although the grafting procedure adopted has demonstrated the inhibition of adverse antibody reactions, the remote possibility of graft rejection cannot be excluded.

CONTRADICTIONS: do not use in patients who have pathologies or conditions that contraindicate surgery or who have an acute or chronic infection (osteomyelitis). Not indicated for patients with immunodeficiency or suffering from autoimmune diseases, particularly children and the elderly.

Do not use directly or near large vessels or nerves. Do not use on poorly vascularized, necrotic or infected receiving sites.

ADVERTENCE: the product is valid for three years from the date of sterilization, which must be closed by repositioning the flap, followed by suturing. The graft site should be well vascularized and the product well stabilized and protected. We recommend you apply a collagen membrane OsteoBiol® Evolution to protect the graft. In oral surgery, the product must be placed in the superobstetral position.

CONTRADICTIONS: do not use in patients who have pathologies or conditions that contraindicate surgery or who have an acute or chronic infection (osteomyelitis). Not indicated for patients with immunodeficiency or suffering from autoimmune diseases, particularly children and the elderly.

Do not use directly or near large vessels or nerves. Do not use on poorly vascularized, necrotic or infected receiving sites.

ADVERTENCE: the product is valid for five years from the date of sterilization and the expiry date is indicated on the label.

STORAGE: The product must be stored away from direct heat sources and sunlight, in a dry and clean place, under the conditions indicated on the label.

DISPOSAL: the product does not contain hazardous substances and it may be disassembled and disposed of as regular municipal waste.

PRODUCT ORIGIN: the last letter on the label of the product code REF indicates the source of the material (S = porcine; E = equine).

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