# Instructions for use

- EN** see [www.fkg.ch](http://www.fkg.ch) for additional information
- FR** informations supplémentaires sur [www.fkg.ch](http://www.fkg.ch)
- DE** weitere Informationen auf [www.fkg.ch](http://www.fkg.ch)

## Finisher

Endodontic instruments  
Instruments endodontiques  
Endodontische Instrumente

FKG REF. 99.X00.10.01B.XX - n°122 - 2021/12

## Finisher

### Description - Beschreibung

25/00

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## Golden rules

Speed : 800-1000 rpm  
Torque : 1 Ncm

- Finisher should be used only after canal preparation to at least ISO 25.
- In multirrooted teeth, begin with the largest canal.
- Work along the entire length of the canal for approximately 1 minute.

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## General information

### Indications for use :

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

The Finisher is used as a final cleaning instrument in root canal treatments.

### Contraindications :

This product contains nickel, and must not be used on individuals with a known allergic sensitivity to this metal.

**Adverse reactions :** None known

### Warnings and precautions :

- Single use instruments must not be retreated for subsequent use. That could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.
- These instruments have not been tested on children, pregnant nor breastfeeding women.

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This is to certify that:

**Emergo Global Consulting, LLC**

2500 Bee Cave Road  
Building 1, Suite 300  
Austin  
Texas  
78746  
USA

Holds Certificate Number:

FS 594789

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 for the following scope:

Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry.

For and on behalf of BSI:

Graeme Tunbridge, Senior Vice President Medical Devices

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 1 of 2

...making excellence a habit.™

Certificate No: FS 594789

Location	Registered Activities
Emergo Global Consulting, LLC 2500 Bee Cave Road Building 1, Suite 300 Austin Texas 78746 USA	Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation and Australian Sponsor, delivered to the medical device industry. AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations.
I.L. Emergo Israel Ltd. Andrei Sakharov 9 Building 25 - 8th floor Matam North Haifa 3190501 Israel	AMAR Registration and License Holder services delivered through Austin, Texas, USA and Matam, North Haifa, Israel locations.
Emergo Europe Westervoortsedijk 60 6827 AT, Arnhem The Netherlands	Consulting services for regulatory and clinical affairs, quality system implementation and maintenance, training and related services, as well as in country representation services, such as European Authorized Representation delivered to the medical device industry.

Original Registration Date: 2013-04-12

Latest Revision Date: 2023-03-15

Effective Date: 2023-04-02

Expiry Date: 2026-04-01

Page: 2 of 2

This certificate was issued electronically and remains the property of BSI and is bound by the conditions of contract.

An electronic certificate can be authenticated [online](#).

Printed copies can be validated at [www.bsigroup.com/ClientDirectory](http://www.bsigroup.com/ClientDirectory)

Issuing Body: BSI Group The Netherlands B.V., John M. Keynesplein 9, 1066 EP Amsterdam, The Netherlands

BSI Group The Netherlands B.V. is registered in The Netherlands under number 33264284 | A Member of the BSI Group Holdings B.V.

Contact Office: 12950 Worldgate Drive, Suite 800, Herndon, VA 20170-6007 USA.