

Smartcord

Do not re-use

Description

Smartcord is a knitted retraction cord, made of 100% cotton for temporary gingival retraction of the gingival margin.

Indications

Smartcord is used to aid in making impressions for fixed prosthetics and is also used for gingival displacement before bonding and/or restorative placement procedures.

Options

Smartcord is available in 6 diameters identified by different colors.

Smartcord 04 (The Finest)	black/blue
Smartcord 03 (Ultra-Fine)	black/gray
Smartcord 02 (Extra-Fine)	brown/yellow

Smartcord 0 (Fine)	purple/light purple
Smartcord 1 (Medium)	blue/light blue
Smartcord 2 (Thick)	green/light green

Composition

100% Cotton yarn 305cm(120inch)

Procedure

1. Select a cord with an appropriate diameter.
2. Cut the cord to the required length with EasyCut and SmartRuler. If necessary, the cord can be impregnated with a hemostatic solution.
3. Insert the cord into the sulcus by means of a retraction cord packer.
4. Leave the cord about 5 minutes in the sulcus.
5. Remove the cord just before taking the impression. Slightly moisten the cord to make its removal easier and reduce gingival hemorrhage.

Precautions

- Check the conditions of gingival tissue (eg. depth or thickness)
- For professional use only.
- For single use only.

- To avoid a possible permanent displacement of the gingiva, the cord should not be remained for longer than 20 minutes in the sulcus.
- After finishing the procedure, make sure to remove the cord from the sulcus.
- The package must be close immediately after use.
- Store in a dry, dark place between 1 °C and 30 °C

Symbols

 CE mark	 Medical Device	 Do not re-use	 Keep dry	 Date of Manufacture	 Keep away from sunlight
 For professional use only	 Temperature limit	 Manufacturer	 Consult instructions for use	 LOT Batch code	 Authorized representative in the European Community
 Manufacture EASTDENT CO., LTD 64, Gugaj-ro, Giheung-gu, Yongin-si, Gyeonggi-do, 16972, Korea Tel : +82-31-693-5767, Fax : +82-31-693-5767			 KTR Europe GmbH Mergenthalerallee 77, 65760, Eschborn, Germany Tel: +49-6196-887-170		
EDSIE, Rev6, 03/2022					