



Information guide

unica anterior



powered by:

STYLE ITALIANO™

matrix
systems





polydentia

swiss manufacture 



"In today's conservative dentistry, the choice of products and the techniques we apply everyday must be predictable and cost-effective, incorporating clear-cut advantages to our daily procedures. Unica is the solution to everyday anterior restorations, whether it be class III, IV, V, shape modifications, and/or composite veneers in direct stratification technique. One effective matrix for everyday anterior aesthetic and functional results."

Dr. Jordi Manauta, DDS
Head of StyleItaliano Idea Factory
Sestri Levante, Italy

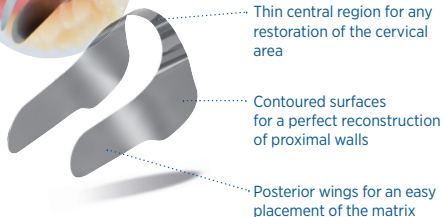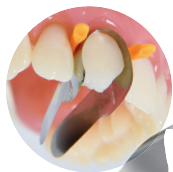

Index

Class III Restoration	3
Class IV Restoration	4
Class V Restoration	5
Composite Veneers	6
Shape modification	7
Refill	8

Class III restoration



By courtesy of
Dr. Dan Lazar
Romania



1. Pre-operative situation



2. The cavities after preparation
(cleaning, caries removal and
disinfection)



3. Positioning of Unica anterior



4. The cavity filled with
composite



5. Post-operative view of the
restoration



*Unica may be cut to fit
only one proximal surface*



for more
information

Class IV restoration



By courtesy of
Dr. Dan Lazar
Romania



for more
information



1. Pre-operative situation



2. Surfaces preparation and
cleaning



3. Positioning of Unica anterior



4. Tooth surface restoration



5. Post-operative view of the
restoration



*Unica may be cut to fit
only one proximal surface*

Class V restoration



By courtesy of
Dr. Giuseppe Chiodera
Italy



1. Pre-operative situation



2. The cavity after preparation
(cleaning, caries removal and
disinfection)



3. Positioning of Unica anterior



4. The cavity filled with
composite



5. Immediate post-operative
view of the restoration



for more
information

Composite veneers



By courtesy of
Dr. Monaldo Saracinelli &
Dr. Antonino Nicolò
Italy



1. Pre-operative situation



2. Surface preparation and
positioning of Unica anterior



3. Tooth surface restoration



4. Clinical situation before
finishing and polishing



5. Post-operative view of the
restoration



for more
information


Shape modification



By courtesy of
Dr. Carlos Fernández Villares
Madrid, Spain



1. Pre-operative situation

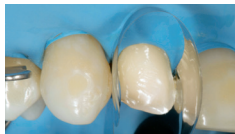

2. Tooth surface preparation



3. Positioning of Unica anterior



4. Tooth surface build up



5. Post-operative view of the
restoration



for more
information








Unica Introkit anterior

REF 6900

- 35 pcs Unica anterior matrices
- 40 pcs myWedge XS
- 40 pcs myWedge S
- 40 pcs myWedge M
- 1 pc myCustom Resin, 3 ml syringe
- 7 pcs needle tips gauge 18 for myCustom Resin
- 1 pc myQuickmatrix Forceps



Refill

	Quantity	REF
 Unica anterior matrices	12	6912
 Unica anterior matrices	50	6950
 myWedge, Extrasmall (XS)	100	7101
 myWedge, Small (S)	100	7102
 myWedge, Medium (M)	100	7103
 myCustom Resin, 3ml syringe & 10 pcs needle tips gauge 18		6811
 myQuickmatrix Forceps		5703

Swiss Made
Polydentia SA
via Cantonale 47
6805 Mezzovico-Vira
Switzerland

phone: +41 91 946 29 48
fax: +41 91 946 32 03

info@polydentia.ch
www.polydentia.ch



Visit "Unica
Community"
to share
Unica anterior
clinical cases

Follow us
#thinkunica

CE REV 01.05.2018

 polydentia
swiss manufacture 